STORZ

KARL STORZ — ENDOSKOPE

 40701620-1 UNIDRIVE® S III ENT

 40701620-1 UNIDRIVE® S III ENT

 40701620-1 UNIDRIVE® S III ENT

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1 Important information for users of KARL STORZ devices

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended as an aid in setting up, connecting and operating the UNIDRIVE® S III ENT motor system with the different application instruments.

- INTRA drill handpiece (2525xx)*
- High-Speed drill handpieces 2526xx**
- Drill (32xxxxxx), (33xxxxxx), (35xxxxxx), (360000x)**
- STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050) with shaver attachment (412xx, 413xx)
- STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050) with sinus burr attachment (413xx)
- DRILLCUT-X® II shaver handpiece (40712050/40712055) with shaver attachment (412xx, 413xx)
- DRILLCUT-X® II shaver handpiece (40712050/40712055) with sinus burr attachment (413xx)
- STAMMBERGER-SACHSE intranasal drill (660000)*
- Micro saw (254000)*
- Micro sagittal saw (254100)*
- Osseoscalpel (254200)*
- Micro compass saw (254300)*
- Dermatom (253xxx)*

*in combination with the EC micromotor II 20711033,

** in combination with the High-Speed micromotor (20712033)

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**Important information for
users of KARL STORZ
devices**

Please read these instructions carefully. Keep this manual in a convenient and conspicuous location and in its protective casing close to the device.

Users of the various application instruments should familiarize themselves with the operating instructions included with the respective devices.

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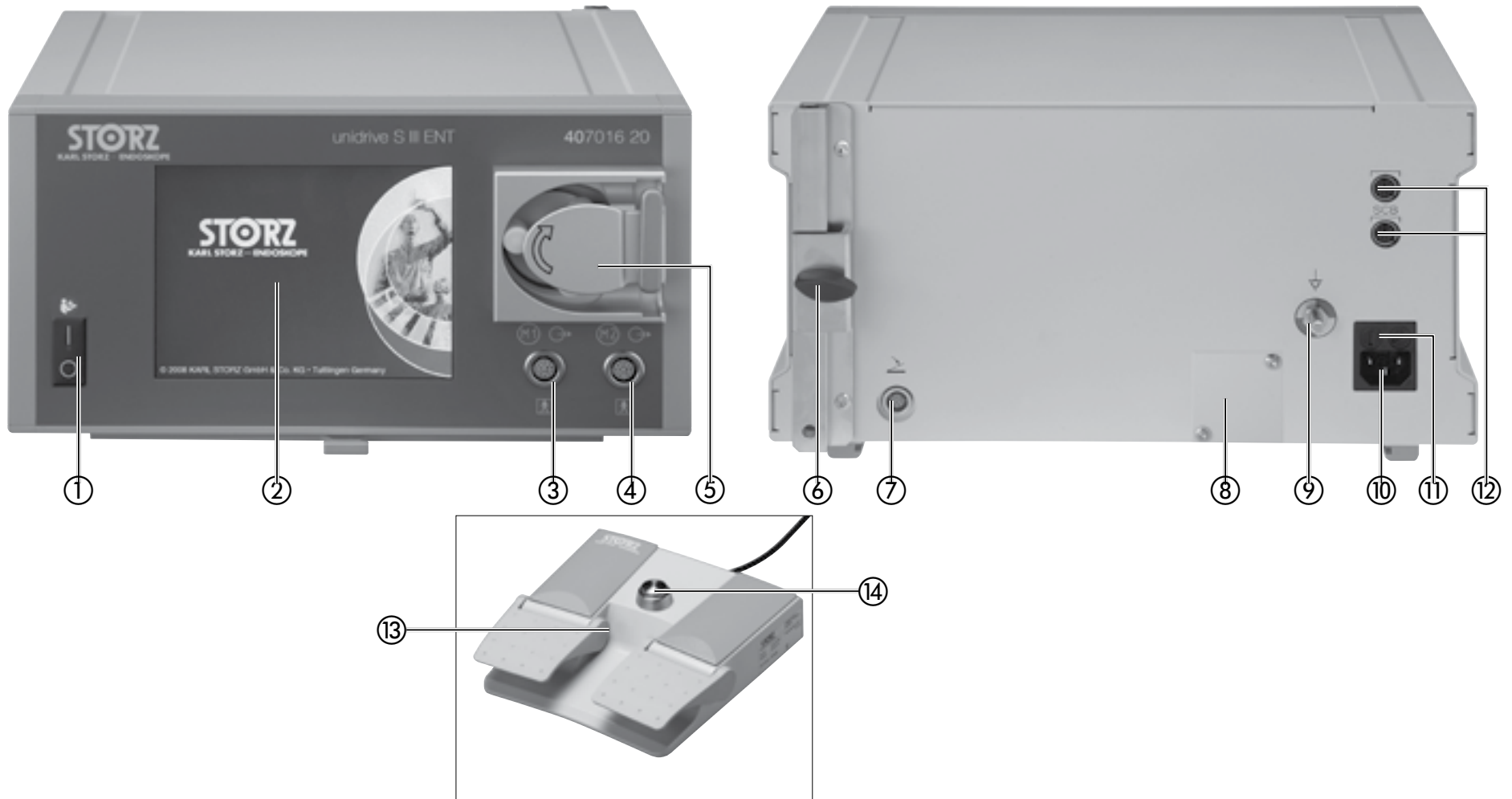
**3 Controls, displays,
connectors, and their uses**

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- ① Power switch for turning the device on and off
- ② TFT touch screen
- ③ Connection socket for motor/handpiece M1
- ④ Connection socket for motor/handpiece M2
- ⑤ Irrigation pump
- ⑥ Holder for irrigator rod
- ⑦ Connection socket for footswitch
- ⑧ USB port (for maintenance only)
- ⑨ Potential equalization connection
- ⑩ Power socket
- ⑪ Fuse holder
- ⑫ ® connection sockets*
- ⑬ Dual-pedal footswitch
- ⑭ Button for changing direction of rotation

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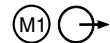
i * **NOTE:** The KARL STORZ SCB interface (KARL STORZ Communication Bus), based on the CAN field bus, permits remote control of equipment functions as well as remote display of equipment parameters.

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Electronic information product pollution control

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Read accompanying documents before use

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DANGER: Risk of explosion if used in the presence of flammable anesthetic gases.

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CAUTION: Do not open the device. Danger of electrical shock!

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Refer service work to qualified service personnel.

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Refer to the instruction manual for sterilization instructions.

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Do not store liquids on or above the device.

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Keep the device out of reach of patients.

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1	Important information for users of KARL STORZ devices	IV
2	Images of the device	VI
3	Controls, displays, connectors, and their uses	VII
4	Symbols employed	IX
5	General information	4
5.1	System description	4
6	Safety instructions	6
6.1	Explanation of warnings and cautions	6
6.2	Intended use	10
6.3	User qualification	10
6.4	Safety precautions at the site of installation	11
6.5	Safety precautions when operating the system	11
7	Operating instructions	12
7.1	Unpacking	12
7.2	Basic equipment	12
7.3	Installing and connecting the device	13
7.4	KARL STORZ SCB	14
7.5	Connecting the footswitch	14
7.6	Connect the application instruments (motor or shaver handpiece)	15
7.7	Coolant and irrigation pump	15
7.8	Preparing the tubing set	16
7.9	Inserting the pump tube	17
7.10	Initial operation	21
7.11	UNIDRIVE® S III ENT function	21
7.12	Selecting the language	22
7.13	Selecting the motor connection	23
7.14	Overview of application instruments	25

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8 Configuring the operating mode	27
8.1 Operating mode: EC micromotor II	27
8.1.1 Application handpiece: Drill 40k	27
8.1.2 Operating mode/application handpiece: Drill 80k	32
8.1.3 Application handpiece: Intranasal Drill	33
8.1.4 Application handpiece: Universal Saws	34
8.1.5 Application handpiece: Micro compass saw	35
8.1.6 Application handpiece: Oscillating Micro Compass Saw	36
8.1.7 Application handpiece: Osseoscalpel	37
8.1.8 Application handpiece: Sagittal Saw	38
8.1.9 Application handpiece: Dermatomes	39
8.2 Operating mode: High-Speed micromotor	40
8.2.1 Application handpiece: Drill 100k	40
8.2.2 Operating mode/application handpiece: Drill 60k	45
8.3 Operating mode: DRILLCUT-X®	46
8.3.1 Sinus Burr	46
8.3.2 Application handpiece: Shaver	50
8.4 Operating mode: DRILLCUT-X® II	51
8.4.1 Application handpiece: Sinus drill	51
8.4.2 Application handpiece: Shaver	55
8.4.3 Application handpiece: Shaver Plus	56
8.5 Using the second motor connection	57
8.6 Settings	58
8.6.1 System information	58
8.6.2 Selecting the language	59
8.6.3 Error messages	60
8.6.4 Date/Time	61
8.6.5 Service	62
8.6.6 Accessory configuration	62

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9	Service and repair.....	64
9.1	Fuse replacement.....	64
9.2	Cleaning, disinfection, and sterilization...	65
9.3	Cleaning, disinfection and sterilization of reusable tubing.....	66
9.3.1	Preliminary cleaning of reusable tubing..	67
9.3.2	Preparation.....	68
9.4	Sterilization.....	70
9.4.1	Steam sterilization.....	70
9.5	Fractionated prevacuum method.....	71
9.6	Maintenance and safety check.....	72
9.6.1	Maintenance.....	72
9.6.2	Safety check/repeat inspection according to IEC 62353.....	72
9.7	Servicing and repair.....	74
9.8	Disposal.....	74
9.8.1	Disposal of Equipment.....	74
9.8.2	Disposal of Tubing Sets.....	74
9.9	Limitation of liability.....	76
9.10	Warranty policy.....	76
10	Technical description.....	77
10.1	General.....	77
10.2	UNIDRIVE® troubleshooting table.....	79
10.3	Troubleshooting.....	81
10.4	Technical data.....	83
10.5	Standard compliance.....	85
10.6	Directive compliance.....	85
10.7	Technical documentation.....	86
10.8	Block diagram.....	87
11	Spare parts, recommended accessories.....	88
11.1	Spare parts.....	88
11.2	Recommended accessories.....	88
12	Appendix.....	93
12.1	Cleaning agents and disinfectants.....	93
12.1.1	Manual preparation.....	93
12.1.2	Machine preparation.....	95
12.2	Electromagnetic Compatibility (EMC) information.....	97

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5 General information

5.1 System description

The new KARL STORZ motor system UNIDRIVE® S III ENT is characterized by the most modern electronics, flexibility, and efficiency.

The system can be applied universally with brushless high-powered electronic drives integrated in the instrument or separately in the ENT range of application.

All settings and functions can be controlled via the integrated touch screen. The settings and functions appear as 'text' in the display.

The preset parameters are saved, i.e. you only have to select them once (exception: CW/CCW, the system always starts with the CW setting).

By simply activating the respective menu field, e.g. device information can be displayed.

Troubleshooting in the OR is easier since any faults that may occur, such as a motor fault, are displayed automatically.

Two connection sockets allow two motors to be simultaneously connected, for example, two drill handpieces in parallel.

Special features

- Automatic detection of connected motors
- Speed range continuously adjustable
- Two motor outputs
- Preselectable max. speed
- Constant high motor power over the entire speed range
- Processor-controlled speed and motor torque
- Integrated coolant and irrigation pump with homogenous irrigation rate over the entire irrigation rate range
- Optimized user control via touch screen
- Controls which are clear and easy to read thanks to a color display
- Pump can be controlled via footswitch

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With KARL STORZ SCB connection option for KARL STORZ Communication Bus (KARL STORZ SCB)

Can be used with:

- INTRA drill handpiece 2525xx
- High-speed handpieces 2526xx
- STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050) with shaver attachment (412xx, 413xx)
- STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050) with sinus burr attachment (413xx)
- DRILLCUT-X® II (40712050/40712055) shaver handpiece with shaver attachment (412xx, 413xx)
- DRILLCUT-X® II (40712050/40712055) shaver handpiece with sinus burr attachment (413xx)
- STAMMBERGER-SACHSE intranasal drill (660000)
- Micro saw (254000)
- Micro sagittal saw (254100)
- Osseoscalpel (254200)
- Micro compass saw (254300)
- Dermatom (253xxx)

i **NOTE:** Please follow the instruction manual enclosed with the respective application instrument!

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Property rights

This product is protected in the USA by (at least one of the following) US Patent(s) 5,788,688; 6,397,286; 6,484,221; 6,824,539.

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6 Safety instructions

Safety instructions are measures intended to protect the user and patients from the risks which could arise through the use of the system.

6.1 Explanation of warnings and cautions

Please read this manual carefully and follow its instructions precisely. The words Warning, Caution, and Note convey special meanings. Wherever they are used in this manual, the accompanying text should be carefully reviewed to ensure the safe and effective operation of the system. To make these words stand out more clearly, they are accompanied by a pictogram.

WARNING: A Warning indicates that the personal safety of the patient or physician may be endangered.

CAUTION: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the device.

NOTE: A Note indicates special information about operating the device or clarifies important information.

WARNING: Read this instruction manual thoroughly before operating the system. In particular, read the chapter on safety instructions to avoid putting your patients, your personnel, or yourself at risk.

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-  **WARNING:** The UNIDRIVE® S III ENT must only be used in medical facilities that conform to the applicable national regulations.
-  **WARNING:** Keep out of reach of patients.
-  **WARNING:** The instruction manuals and interface specifications for medical devices and/or system components used in combination must be observed precisely.
-  **WARNING:** Combinations of medical devices are only assured to be safe if:
- they are identified as such in the respective instruction manual; or
 - the intended use and the interface specifications of the devices used in combination permit this (cf. IEC 60601-1-1).
-  **WARNING:** Connect the device to a properly grounded 'Hospital grade' or 'Hospital use' electrical outlet. Check grounds regularly. Routinely inspect electrical plug and cord. Do not use if inspection reveals damage.
-  **WARNING:** Test this equipment prior to each surgical procedure to ensure that it functions correctly.
-  **WARNING:** Risk of explosion if used in the presence of flammable anesthetic gases.
-  **WARNING:** The UNIDRIVE® S III ENT, in conjunction with the application instruments to be used, is only designed for intermittent operation (50%, t = 1 min). (see Technical Data on page 83).

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WARNING: Always unplug the device from the mains before performing any maintenance or cleaning work on it.

WARNING: Do not open device! Risk of electric shock. Arrange for service work to be carried out by the manufacturer or by personnel authorized by the manufacturer only. Removal of the cover by non-authorized personnel will void the warranty.

WARNING: Always check before and during use whether the desired channel or operating mode has been set.

WARNING: If motor does not stop when desired, turn off the device.

WARNING: Limits can be adjusted when working with RUI. Make sure to check the parameters before and after use when you have worked with RUI.

WARNING: When using rotating instruments, ensure that there is sufficient irrigation and thus cooling. Insufficient irrigation can lead to the instruments becoming smeared, blocked or jerky movements. This can result in increased heat generation and put the patient at risk. Please ensure that there is always sufficient irrigation.

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CAUTION: The device may only be operated at the voltage stated on the rating plate.

CAUTION: When replacing fuses, use only fuses of the same rating.

CAUTION: The UNIDRIVE® S III ENT may only be used with accessories and disposable items which have been designated for use by KARL STORZ.

CAUTION: Do not store liquids on or above the device. Avoid allowing liquids to enter the device.

CAUTION: If the motor becomes blocked for any length of time, it can lead to overheating and cause damage to the motor. If the blockage cannot be disengaged using oscillating operation (if possible), the motor must be shut off and the blockage must be removed manually.

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NOTE: Any damage to the device resulting from incorrect operation is not covered by the manufacturer's warranty.

NOTE: For the cleaning/disinfectant solution, use chemicals that have been specially approved by KARL STORZ for the preparation of medical devices. The current lists of chemicals can be found on the Internet at www.karlstorz.com under 'Hygiene'.

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Before using the UNIDRIVE® S III ENT motor system on a patient, it is imperative that the instructions in this manual are understood and are followed carefully.

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6.2 Intended use

The UNIDRIVE® S III ENT motor system is universally applicable in conjunction with the respective, suitable application instruments for ENT interventions.

The exact 'Intended use' of the application instruments is detailed in the respective instruction manual provided with the application instrument.

The UNIDRIVE® S III ENT motor system may only be used with accessories, parts and disposable items that have been designated by KARL STORZ as suitable for the device.

Unauthorized conversions or modifications to the device are not allowed for safety reasons.

The ENT motor control unit is used for the regulation and control of motors or handpieces and accessories during minimally invasive interventions in the field of ENT for the excision, morcellation, removal, ablation, and transection of bones/tissue.

6.3 User qualification

The UNIDRIVE® S III ENT motor system in conjunction with an application instrument may only be used by physicians and/or medical assistants who have a corresponding specialized qualification and who have been instructed in the use of this device/instrument.

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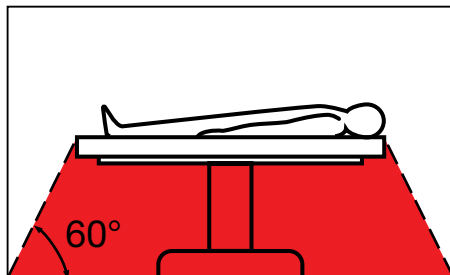
6.4 Safety precautions at the site of installation

The UNIDRIVE® S III ENT motor system may only be used in medical rooms whose electrical systems have been installed in accordance with applicable national regulations.

The UNIDRIVE® S III ENT is equipped with a connector for attaching a ground wire. It should be connected up in accordance with the applicable national regulations.

The UNIDRIVE® S III ENT motor system is not intended for use in hazardous areas. This means, among other things:

wherever easily combustible and explosive inhalation anesthetics and their mixtures are used, the device may not be operated in the demarcated hazard zone. This also applies for easily combustible and explosive chemicals, e.g., skin disinfectants and fast-acting surface disinfectants.

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6.5 Safety precautions when operating the system

WARNING: It is the user's responsibility to make sure the system is safe and operates properly before using it.

During treatment with the UNIDRIVE® S III ENT, the patient must be treated and kept under observation with the usual medical care. This includes keeping a check on the progress of treatment, as well as monitoring the vital signs and the anesthetic.

This device has been tested and is found to comply with the EMC limit values as per EN/IEC 60601-1-2:2001 CISPR 11 Class B.

Observe the information on electromagnetic compatibility in the appendix (pages 97–109).

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7 Operating instructions

7.1 Unpacking

Carefully remove the device and the accessories from the packaging.

Check for missing items and for evidence of shipping damage.

Please file any complaints with the manufacturer or supplier immediately.

If possible, retain the original packing materials for later use; these can be used if the device has to be transported.

7.2 Basic equipment

KARL STORZ set 407016 01-1

- 1 UNIDRIVE® S III ENT 407016 01-1
- 1 Power cord 400 A
- 1 Dual-pedal footswitch with proportional function (200166 30)
- 1 Tubing set for irrigation, can be sterilized, 207116 40
- 1 Clip set 207116 21
- 1 3 pcs. mtp Tubing Sets, single use, sterile 031131-03
- 1 Irrigator rod 207116 80
- 1 SCB connecting cord, length 100 cm, 200901 70
- 1 Instruction manual 96056025D

i NOTE: *Optional accessories can be found under 'Recommended Accessories'.*

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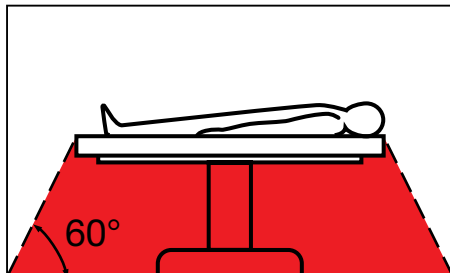
7.3 Installing and connecting the device

NOTE: The UNIDRIVE® S III ENT including the accessories connected may be used only in medical rooms with electrical installations conforming to the applicable national, state, and local electrical regulations.

! WARNING: The UNIDRIVE® S III ENT is not intended for use in potentially explosive atmospheres. Do not operate the system within the hazard zone shown in the diagram while explosive anesthetic gases are in use.

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Place the UNIDRIVE® S III ENT on a level surface. The UNIDRIVE® S III ENT is equipped with a connector for attaching a ground wire ⑨. The device's ground line should be installed by a qualified electrician.

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Before connecting the UNIDRIVE® S III ENT to the electrical supply line, verify that the line voltage stated on the device's identification plate is the same as that of the electrical supply line to be used.

Insert the mains cord into the device's receptacle ⑩, pushing it firmly into place.

! WARNING: Only insert the power plug into and remove it from electrical outlets located outside areas subject to explosion hazards.

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7.4 KARL STORZ SCB

i **NOTE:** To prevent the SCB connecting cable from being pulled out accidentally, the SCB connector possesses a protection device.

Pull back the protection device of the SCB connector and insert the connector into one of the SCB sockets ⑫. Connect the other end of the cord to the KARL STORZ SCB (KARL STORZ Communication Bus) control unit or other SCB devices (see KARL STORZ SCB System Instruction Manual).

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7.5 Connecting the footswitch

To use the footswitch ⑬, insert the connecting cord* of the footswitch into the socket for the footswitch ⑦ on the rear of the UNIDRIVE® S III ENT. Align the red dot on the connecting cord with the red dot on the footswitch socket.

i **NOTE:** If no footswitch is connected when the device is switched on, the message: 'E11: NO FOOTSWITCH' flashes in the status line of the display (see arrow).

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7.6 Connect the application instruments (motor or shaver handpiece)

Prepare the application instrument according to the instruction manual.

i NOTE: Please follow the instruction manual enclosed with the application instrument!

The connecting cable* coming from the application instrument (EC micromotor or shaver handpiece) must be connected to motor output M1 ③ and/or M2 ④.

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The connection will be made properly when the red dot on the connecting cord is aligned with that on the socket.

i NOTE: If no 'motor' is connected when the device is switched on, the message: 'E12: NO MOTOR' flashes in the status line of the display.

i * NOTE: To unplug the connecting cable, pull out the plug — do not pull directly on the cable!

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7.7 Coolant and irrigation pump

The UNIDRIVE® S III ENT control unit is equipped with an integrated roller irrigation pump which takes on the function of a coolant pump as well.

i NOTE: Use physiological saline solution as a coolant and irrigation fluid.

The pump function is available with the following application instruments: SHAVER, DRILLCUT-X® II, DRILLCUT-X®, DRILL, SINUS BURR, INTRANASAL and SAWS.

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If required, mount the irrigator rod

If required, insert the irrigator rod into the holder ⑥ in order to hang up an irrigation bag or an irrigation bottle. Lock the irrigator rod by turning the knob on the holder (in a clockwise direction).

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If required, hang the irrigation bag or irrigation bottle on the irrigator rod.

CAUTION: Mount the irrigator rod in such a way that the irrigation bag neither hangs directly above the device nor touches the device (heat emission) (see section 'Safety instructions').

NOTE: If an irrigation bottle is used, please ensure that the bottle is aerated. Otherwise irrigation performance decreases.

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7.8 Preparing the tubing set

NOTE: The tubing set is not sterile when delivered. For sterilization, see pages 65 et seq.

NOTE: The tubing is delivered ready-assembled. If the tubing has been disassembled, for preparation for example, it must be reassembled as described below.

Push/slide both ends of the pump tube onto the connectors until they click into place.

Push/slide the inflow tubing onto the narrow pump tubing connector (large diameter). Push/slide the outflow tubing onto the wide pump tubing connector (small diameter).

After attaching the pump tubing, press the movable clamp ring on the wide pump tubing connector against the pump tubing as far as it will go.

WARNING: It is essential to carry out a leakage test every time before you attach the pump tubing to the device. See page 69.

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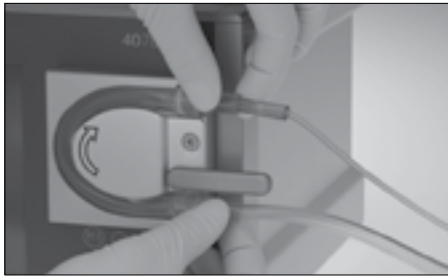
7.9 Inserting the pump tube

Flip the lever for retracting/extending the pump roll down into the horizontal position.

! **CAUTION:** Use only the tubing sets intended for use with the UNIDRIVE® S III ENT.

i **NOTE:** The pump tubing connectors have different widths so that the pump tubing can only be inserted in one direction. There is also a directional arrow on the connector to the drainage tubing.

Attach the pump tubing to the roller pump and press both connectors into the fastening grooves.

i **NOTE:** If the connectors cannot be pressed into the fastening grooves, they have not been attached correctly to the tubing. If necessary, attach the connectors on the pump tubing more securely.

i **NOTE:** As the pump tubing may develop a leak due to application wear, we recommend changing the pump tubing after 5 uses.

! **CAUTION:** If the pump tubing is perforated, there is a danger of fluid entering the device!

Move the lever for retracting/extending the pump roller into the vertical position and the pump tubing is locked in place.

i **NOTE:** If the pump tube is not locked in place, the pump cannot be started, error message "E16: IRRIGATION PUMP NOT CLOSED" is shown.

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Removing the tubing set

Press lever down to the horizontal position.
Remove the pump tube from the reel pump.

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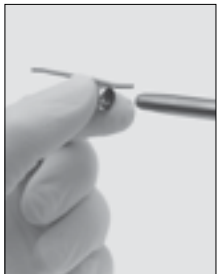
The inflow tubing of the silicone tubing set for irrigation is equipped with a puncture needle and a tubing clamp.
Close the tubing clamp.
Insert a piercing punch into the intended position on the irrigation bag or irrigation bottle.

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If necessary, attach a cooling/irrigation channel onto the drill handpiece/microsaw (see arrow).
Attach the other end of the silicone tubing to the irrigation connection of the DRILLCUT-X® shaver handpiece, the drill handpiece or the microsaw.
Using clips from the clip set (Art. No. 20711021), attach the silicone tube to the DRILLCUT-X® and DRILLCUT-X® handpiece/micro saw and/or connecting cord.

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Disposable tubing set (mtp Tubing Set 031131-01)

Insert the tubing into the holder.
Connect the relevant tubing ends with the irrigation liquid bag (puncture needle) or attach to the irrigation connector of the DRILLCUT-X® shaver handpieces, drill handpiece or the microsaw.

i NOTE: If an irrigation bottle is used, open the ventilation cap at the puncture needle in order to ventilate the bottle.

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Irrigation pump control

i NOTE: Operation of the the pump functions via touch screen is described in the respective section under 'Initial operation'.

The pump function is disabled/enabled via the 'Pump' operating field on the touch screen of the control unit. If the pump function is enabled (pump button is orange) the pump is automatically activated upon actuation of the right footswitch pedal ...

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... together with the motor, and deactivated when the pedal is released.

i NOTE: Activating the left-hand footswitch pedal for a longer period of time activates the irrigation pump alone (if it was activated previously).

The pump runs backwards for a short time after deactivating to aspirate the irrigation solution. This prevents any liquid from dripping.

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Setting the irrigation flowrate - Touch screen

The '+/-' buttons in the touch screen's 'Pump' screen can be used to set the specified irrigation flowrate for the pump in the respective operating mode in 9 pump levels.

The currently selected pump level is displayed accordingly.

NOTE: If the pump level is set to '0', the pump does not start.

Setting the flowrate – Footswitch

NOTE: The irrigation flowrate can also be set using the footswitch.

Pressing the left footswitch pedal quickly raises or lowers (depending on setting) the irrigation flowrate level by level.

NOTE: Setting with the left foot pedal is only possible if the right foot pedal is not active; i.e. not while drilling or while operating the motor/shaver.

NOTE: The last value that was set for the solution flowrate while in the defined operating mode is retained even after switching the control unit off.

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7.10 Initial operation

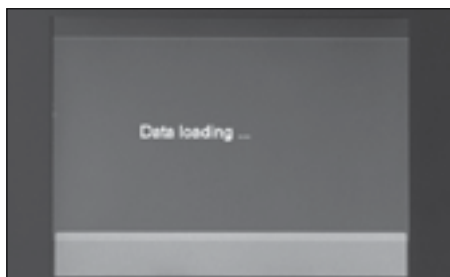
Turn on the UNIDRIVE® S III ENT at the power switch ① (power switch lights up).

i NOTE: Before switching the device on, the user must connect the footswitch and an application instrument to the device. This is checked by the software and if necessary indicated by a corresponding error message in the bottom image margin.

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Conecte el UNIDRIVE® S III ENT pulsando el interruptor de red ① (se enciende el interruptor de red).

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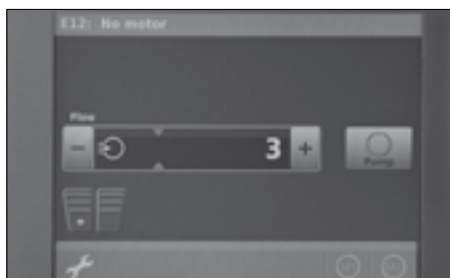
The screen shown on the left appears for the time it takes the application to start.

i NOTE: The next pages contain a description of the simplest application example; adopting the most recently used mode, as well as the adjustments for speed and pump settings. During the switch-on and switch-off procedures, a single signal tone is heard.

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7.11 UNIDRIVE® S III ENT function

A start screen is displayed once the device has started. The function only becomes active once application instruments have been connected with the device.

! CAUTION: Do not use objects in order to avoid damaging the sensitive touch screen surface.

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On the screen shown, the start screen with connected EC micromotor is displayed.

i NOTE: In the status line of the run screen error messages are shown as text, where relevant (flashing text).

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7.12 Selecting the language

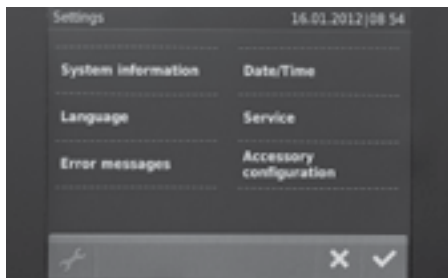
It may be necessary to change the display language of the run screen as English is the display language upon delivery.

This is done by pressing the 'menu' symbol (wrench) in the bottom left.

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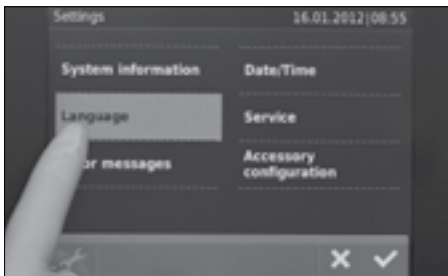
Press the 'LANGUAGE' button (2nd button from the top).

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Select the desired language*.

Select one of the ten available languages by pressing the respective language button. The languages currently available are displayed over two screens.

- ✘ = Cancel selection
- ✓ = Confirm selection

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7.13 Selecting the motor connection

Various application instruments operated within various speed ranges can be connected up to the control unit. To do so, the required instrument can be optionally connected to the motor connection socket (M1 and M2). Having detected the handpiece automatically, the UNIDRIVE® S III ENT loads the standard parameters and displays them on the screen. The symbol for the respective instrument is displayed in the bottom right of the screen. There are two connection sockets available, which can be used with different application instruments.

The motor connection socket can be selected via the buttons for the respective application in the lower right of the screen.

It is only possible to switch between the individual motor connection sockets when both channels are occupied.

i NOTE: It is also possible to switch between M1 <-> M2 by pressing the button **14** on the footswitch for longer (> 2 s) (see also 'Footswitch function', p. 30).

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7.14 Overview of application instruments

INTRA drill handpiece (2525xx) with EC micromotor II (20711033) and drill 26xxxx, 649xxx K, 649xxx GK, 649xxx GL, 649xxx L)	Page 26	DRILL	depending on handpiece and setting 1,000–40,000 rpm and/or 1,000–80,000 rpm
Intranasal Drill (660000) with EC Micro Motor II (20711033)	Page 32	INTRANASAL	3.000–60.000 rpm
High-Speed micromotor (20712033) with High-Speed handpieces (2526xx), drills (32xxxxxx), (33xxxxxx), (35xxxxxx)	Page 39–44	DRILL	depending on handpiece and setting 1,700–60,000 rpm and/or 1,700–100,000 rpm
DRILLCUT-X® II shaver handpiece (40712050/40712055) with shaver attachment (412xx, 413xx)	Page 54	DRILLCUT-X® II SHAVER	300–10,000 rpm Preset = 4,000 rpm
DRILLCUT-X® II shaver handpiece (40712050/40712055) with sinus burr attachment (413xx)	Page 50	DRILLCUT-X® II SINUS BURR	300–12,000 rpm Preset = 12,000 rpm
STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050)	Page 49	DRILLCUT-X® SHAVER	300–7,000 rpm Preset = 7,000 rpm
STAMMBERGER-CASTELNUOVO DRILLCUT-X® shaver handpiece (40711050) with sinus burr attachment (413xx)	Page 45	DRILLCUT-X® SINUS BURR	800–12,000 rpm Preset = 12,000 rpm
Microsaw (254000)*, micro sagittal saw (254100)*, osseoscalpel (254200)*, micro compass saw (254300)* * with EC micromotor II (20711033)	Page 34	SAWS	1,000–20,000 rpm presetting dependent on handpiece 15,000 rpm
Dermatome (253xxx) with EC micromotor II (20711033)	Page 38	DERMATOMES	1,000–8,000 rpm

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8 Configuring the operating mode

After switching on and enabling the function of the UNIDRIVE® S III ENT, the adjacent screen is displayed.

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8.1 Operating mode: EC micromotor II

(INTRA drill handpiece 2525xx with EC micromotor II 20711033)

To activate the 'EC micromotor II' operating mode, connect the EC micromotor II up to the UNIDRIVE® S III ENT using the connecting cord (as of V1.05).

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Select the 'Drill 40k' application.

All applications operated with the EC micromotor II are available for selection here.

These are: drill 40k, drill 80k, intranasal drill, universal saws, and dermatomes.

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The term ‘Drill 40k’ can now be seen in the status display (top left).

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Once the confirm button has been pressed, the ‘Drill 40k’ function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section ‘Configuring the operating mode’ as of page 26.

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**Configuring the operating mode:
Activating the ‘SOFT START’ option**

The option ‘SOFT START’ makes it possible to accelerate the drill handpiece continuously to the default maximum speed when activating the foot-switch pedal within a predefined amount of time.

Press the ‘SOFT START’ button in the top right of the screen.

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The chosen settings will be saved when the required time is pressed.

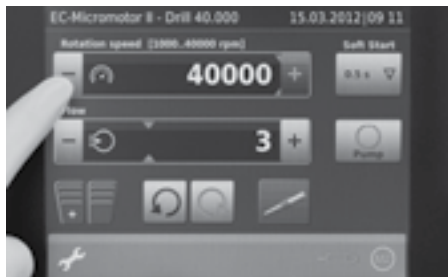
You are then returned directly to the application screen.

i NOTE: The selected 'ramp' (start-up behavior) is used for the respective function. It is also generally possible to use different ramps for different applications.

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**Configuring the operating mode:
Preselecting the maximum speed (setpoint value)**

The top field of the run screen is used for setting and displaying the motor speed. The available speed range is shown in the left-hand bargraph display at the top and bottom (in this case 1,000 to 40,000 rpm).

Use the '+/-' buttons' to define the maximum speed (setpoint value). This preset speed is displayed by the orange bar and figures.

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i NOTE: Please observe the maximum permitted rpm for the selected attachment!

If the EC micromotor II is switched on via the right footswitch, the bargraph display for the speed turns light blue and displays the current speed (actual value).

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**Configuring the operating mode:
Configuring the coolant and irrigation pump**

The integrated coolant and irrigation pump can be optionally activated in all operating modes (apart from 'DERMATOMES'). To activate, press the 'PUMP' button in the center of the screen.

i NOTE: Information on adjusting the irrigation pump settings can be found on page 18.

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If the pump is activated, the button and the bargraph display are also activated.

The irrigation flowrate can be preset using the '+/- buttons' (setpoint value).

The setpoint value is displayed by the orange line in the bargraph display.

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If the handpiece is started with the right-hand footswitch, the pump starts automatically and supplies the preset flow quantity.

The left-hand footswitch pedal (actuation >2 s) only starts the pump and the irrigation solution is supplied with the preset irrigation flow. The handpiece is not activated.

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Footswitch function

The footswitch has the following functions in the 'DRILL' operating mode:

a.) Activate the left pedal longer than 2 seconds

Only the pump is activated (irrigation), irrigation flow rate according to preset value (see 'Irrigation pump control', p. 18)

b.) Press left pedal briefly

The irrigation flowrate can be increased or reduced with the left pedal as of V1.03 (depending on the setting in the 'Accessories configuration' menu).

c.) Activate right pedal

This pedal has a proportional function (gas pedal function), the speed is varied depending on the pedal position. The handpiece is started, pump runs with the defined irrigation flowrate.

d.) Press yellow button (top) briefly

Changes the direction of rotation.

i **NOTE:** The standard setting after each time the device is switched on or selected is Clockwise ↻.

e.) Press the yellow button for longer than 2 sec.

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8.1.2 Operating mode/application handpiece: Drill 80k

To change the operating mode (selection of application handpiece), press the button of the displayed attachment on the right of the screen.

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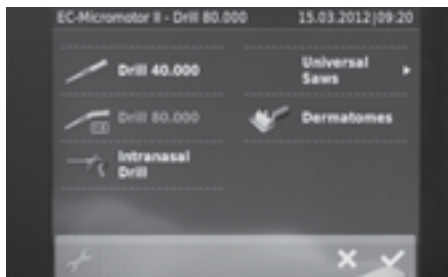
Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Drill 80k' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Drill 80k' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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8.1.3 Application handpiece: Intranasal Drill

To change the operating mode (selection of application handpiece), press the button of the displayed attachment on the right of the screen.

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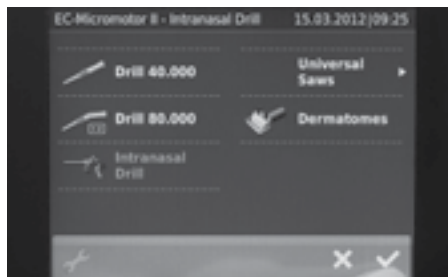
Select the 'Intranasal Drill' application.

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Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Intranasal Drill' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Intranasal Drill' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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8.1.4 Application handpiece: Universal Saws

To change the operating mode (selection of application handpiece), press the button of the displayed attachment on the right of the screen.

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Select the 'Universal Saws' application.
Pressing the confirm button allows you to proceed to the 'Universal Saws' submenu.

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The submenu allows you to choose from the saws on offer.

The following saws are available to you:

- Micro compass saw (as of page 34)
- Oscillating micro compass saw (as of page 35)
- Osseoscalpel (as of page 36)
- Sagittal saw (as of page 37)

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8.1.5 Application handpiece: Micro compass saw

You are now in the 'Universal Saws' submenu.

i NOTE: Information on how to access this submenu can be found on p. 33.

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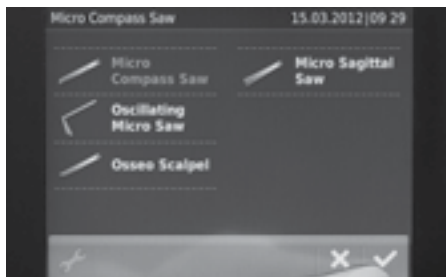
Select the 'Micro Compass Saw' application.

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Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Micro Compass Saw' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Micro Compass Saw' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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**8.1.6 Application handpiece:
Oscillating Micro Compass Saw**

You are now in the 'Universal Saws' submenu.

i NOTE: Information on how to access this submenu can be found on p. 33.

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Select the 'Oscillating Micro Compass Saw' application.

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Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Oscillating Micro Compass Saw' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Oscillating Micro Compass Saw' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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**8.1.7 Application handpiece:
Osseoscalpel**

You are now in the 'Universal Saws' submenu.

NOTE: Information on how to access this submenu can be found on p. 33.

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Press the confirm button in the bottom right to be able to use the selected function.

NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Osseoscalpel' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Osseoscalpel' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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8.1.8 Application handpiece: Sagittal Saw

You are now in the 'Universal Saws' submenu.

i NOTE: Information on how to access this submenu can be found on p. 33.

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Select the 'Sagittal Saw' application.

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Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Sagittal Saw' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Sagittal Saw' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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**8.1.9 Application handpiece:
Dermatomes**

To change the operating mode (selection of application handpiece), press the button of the displayed attachment on the right of the screen.

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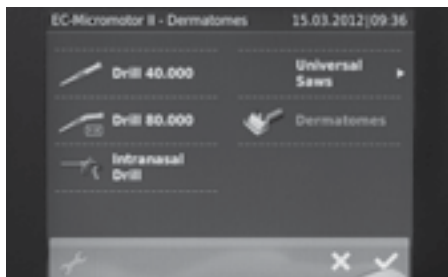
Select the 'Dermatomes' application.

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Press the confirm button in the bottom right to be able to use the selected function.

i NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Dermatomes' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Dermatomes' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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**8.2 Operating mode:
High-Speed micromotor**

High-Speed drill handpiece 2526x with High-Speed micromotor 20712033

To activate the 'High-Speed micromotor' operating mode, connect the High-Speed micromotor up to the UNIDRIVE® S III ENT using the connecting cord (as of V1.07).

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Select the 'Drill 100k' application.

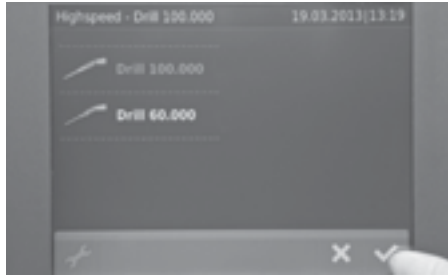
All applications operated with the High-Speed micromotor are available for selection here.

These are: Drill 100k, drill 60k, perforator and craniotome.

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Press the confirm button in the bottom right to be able to use the selected function.

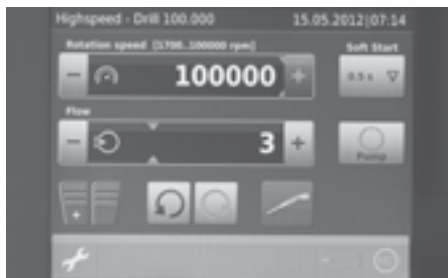
❗ NOTE: Once an application has been selected, the display can be returned to the run screen using the confirm button.

The term 'Drill 100k' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Drill 100k' function is available. The correct settings for this function are loaded automatically.

To adjust the individual settings in this mode, please proceed as described in the section 'Configuring the operating mode' as of page 26.

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**Configuring the operating mode:
Activating the 'SOFT START' option**

The option 'SOFT START' makes it possible to accelerate the drill handpiece continuously to the default maximum speed when activating the footswitch pedal within a predefined amount of time.

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The top field of the run screen is used for setting and displaying the motor speed. The available speed range is shown in the left-hand bargraph display at the top and bottom (in this case 1,000 to 100,000 rpm).

Use the '+/- buttons' to preselect the maximum speed (setpoint value). This preset speed is displayed by the orange bar and figures.

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i NOTE: If the High-Speed micromotor is switched on via the right footswitch, the bargraph display for the speed turns light blue and displays the current speed (actual value).

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The integrated coolant and irrigation pump can be optionally activated in all operating modes. To activate, press the 'PUMP' button in the center of the screen.

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b.) Press left pedal briefly

The irrigation flow rate can be increased or reduced with the left pedal as of V1.03 (depending on the setting in the 'Accessories configuration' menu).

c.) Activate right pedal

This pedal has a proportional function (gas pedal function), the speed is varied depending on the pedal position. The handpiece is started, pump runs with the preset irrigation flow rate.

d.) Press button  at top briefly

Changes the direction of rotation.

 **NOTE:** The standard setting after each time the device is switched on or selected is Clockwise .

e.) Press button  at top for longer than 2 sec.

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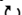
8.2.2 Operating mode/application hand-piece: Drill 60k

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The term 'Drill 60k' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Drill 60k' function is available. The correct settings for this function are loaded automatically.

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8.3 Operating mode: DRILLCUT-X®

To activate the 'DRILLCUT-X®' operating mode, connect the DRILLCUT-X® up to the UNIDRIVE® S III ENT using the connecting cord.

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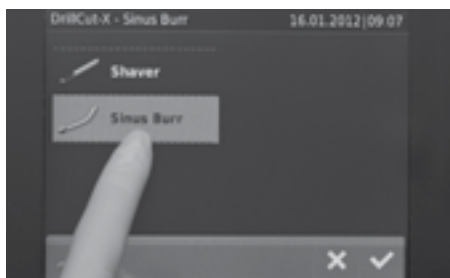
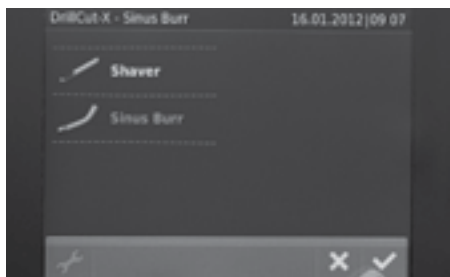
Select the 'Sinus Burr' application.

All applications operated with the DRILLCUT-X® are available for selection here. These are: Sinus burr, shaver.

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The term 'Sinus Burr' can now be seen in the status display (top left).

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The option 'SOFT START' makes it possible to accelerate the drill handpiece continuously to the default maximum speed when activating the foot-switch pedal within a predefined amount of time.

Press the 'SOFT START' button in the top right of the screen.

The chosen settings will be saved when the required time is pressed. You are then returned directly to the application screen.

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The top field of the run screen is used for setting and displaying the motor speed. The available speed range is shown in the left-hand bargraph display at the top and bottom (in this case 300 to 10,000 rpm). The maximum speed (setpoint value) can be preset using the '+/-' buttons'. This preset speed is displayed by the orange bar and figures in the bargraph display.

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If the 'handpiece' is switched on via the right foot-switch pedal, the bargraph display and the figures for the speed turn light blue and display the current speed (actual value).

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The integrated coolant and irrigation pump can be optionally activated in all operating modes (apart from 'DERMATOMES'). For the activation, the 'PUMP' button on the right of the run screen must be clicked.

i NOTE: Information on adjusting the irrigation pump settings can be found on page 26.

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The irrigation flowrate can be preset using the '+/-' buttons' (setpoint value).

The setpoint value is displayed by the blue line in the bargraph display.

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Footswitch function

The footswitch has the following functions in the 'SINUS BURR/SHAVER' operating mode:

- a) Activate left pedal**
Only the pump is activated (irrigation), irrigation flow rate according to preset value (see 'Irrigation pump control', p. 18)
- b) Press left pedal briefly**
The irrigation flow rate can be increased or reduced with the left pedal as of V1.03 (depending on the setting in the 'Accessories configuration' menu).
- c) Activate right pedal**
This pedal has a proportional function (gas pedal function), the speed is varied depending on the pedal position. The handpiece is started, pump runs at the preset irrigation flowrate.
- d) Press button ⑭ at top for longer than 2 sec.**
Switches from motor output 1 (M1) to motor output 2 (M2), or vice versa.
- e) Press left and right pedal simultaneously**
Pushing both pedals simultaneously allows you to switch between the shaver and sinus burr modes. A signal tone is emitted when you switch between modes.

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The term 'Shaver' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Shaver' function is available. The correct settings for this function are loaded automatically.

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8.4 Operating mode: DRILLCUT-X® II

To activate the 'DRILLCUT-X® II' operating mode, connect the DRILLCUT-X® II up to the UNIDRIVE® S III ENT using the connecting cord.

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8.4.1 Application handpiece: Sinus drill

In order to be able to work with the selected application handpiece, please insert the device into the selected motor connection socket. The handpiece is detected automatically and the corresponding screen loaded.

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Select the 'Sinus drill' application.

All applications operated with the DRILLCUT-X® are available for selection here. These are: sinus drill, shaver.

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The term 'Sinus drill' can now be seen in the status display (top left).

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- a) Activate left pedal**
Only the pump is activated (irrigation), irrigation flow rate according to preset value (see 'Irrigation pump control', p. 18)
- b) Press left pedal briefly**
The irrigation flow rate can be increased or reduced with the left pedal as of V1.03 (depending on the setting in the 'Accessories configuration' menu).
- c) Activate right pedal**
This pedal has a proportional function (gas pedal function), the speed is varied depending on the pedal position. The handpiece is started, pump runs at the preset irrigation flow rate.
- d) Press button at top for longer than 2 sec.**
Switches from motor output 1 (M1) to motor output 2 (M2), or vice versa.
- e) Press left and right pedal simultaneously**
Pushing both pedals simultaneously allows you to switch between the shaver and sinus burr modes. A signal tone is emitted when you switch between modes.

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8.4.3 Application handpiece: Shaver Plus

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i NOTE: The Shaver/Shaver Plus modes can be set in the configuration (Accessory configuration). Depending on the setting, the Shaver or Shaver Plus handpiece appears in the selection menu.

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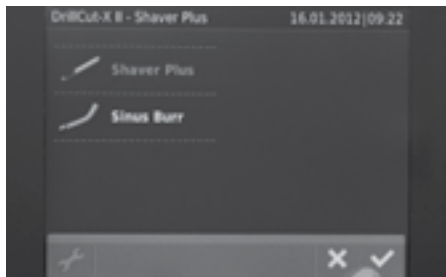
Select the 'Shaver Plus' application.

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The term 'Shaver Plus' can now be seen in the status display (top left).

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Once the confirm button has been pressed, the 'Shaver Plus' function is available. The correct settings for this function are loaded automatically.

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8.5 Using the second motor connection

Since the UNIDRIVE® S III ENT has two motor connections (channels), a second application handpiece can be connected.

The configurations and settings described in this instruction manual apply for both connections (M1 and M2). The two motors can be configured independently of one another.

To switch between the motor output 1 (M1) and motor output 2 (M2), the corresponding button can be selected in the display (bottom right) or the button **14** on the footswitch must be pressed for longer than 2 sec.

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8.6 Settings

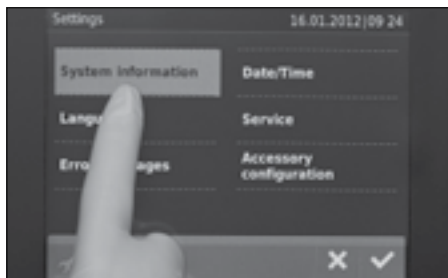
8.6.1 System information

To call up the menu functions, press the 'Menu' button (wrench) in the bottom right.

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This takes you to the 'MENU' screen (image on the left). Press 'SYSTEM INFORMATION' for detailed information on your UNIDRIVE® S III ENT.

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In the upper area the device number (serial number) is displayed.

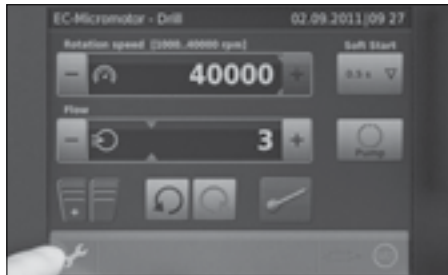
The operating hours and the software version statuses are displayed in the lower section.

Pressing the 'OPERATING MODE' button takes you back to the run screen from which the MENU was called up. This applies for all menu levels.

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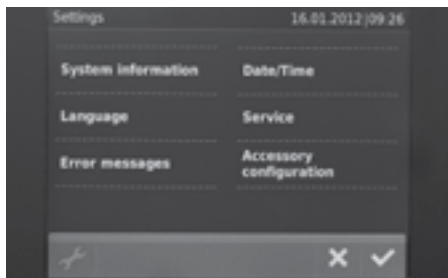
This is done by pressing the 'Menu' symbol (wrench) in the bottom left.

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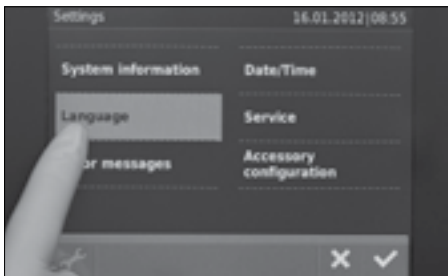
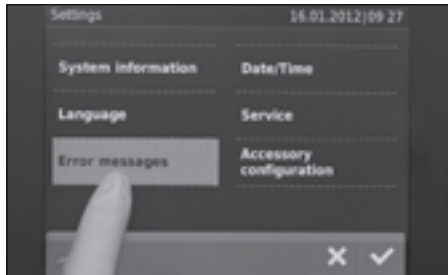
Select the desired language*.
Select one of the ten available languages by pressing the respective language button. The languages currently available are displayed over two screens.

✘ = Cancel selection
✔ = Confirm selection

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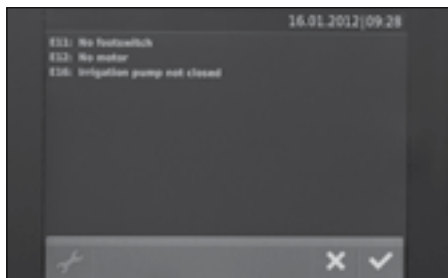
8.6.3 Error messages

In the 'MENU', you can enter the 'ERROR MESSAGE' submenu by pressing the 'ERROR MESSAGE' button.

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In the upper area, the device number (serial number) is displayed. In the lower area, all of the error messages which occur appear in order of importance.

E01-E10: Hardware error messages, which are potentially caused by a device fault.

E11-E30: User error messages, which are due to incorrect configuration of the device (e.g. the footswitch is not connected). See Error table on p. 79.

The example on the left displays three error messages:

E11: NO FOOTSWITCH (no footswitch is connected)

E12: NO MOTOR IN M1 (no motor/handpiece connected to M1)

E16: IRRIGATION PUMP NOT CLOSED (the pump head is not closed)

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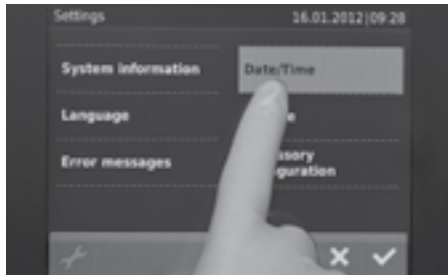
8.6.4 Date/Time

In the 'MENU', you can enter the 'DATE/TIME' submenu by pressing the 'DATE/TIME' button.

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Date settings

Under 'SET DATE' you can perform settings concerning the date format and the current date.

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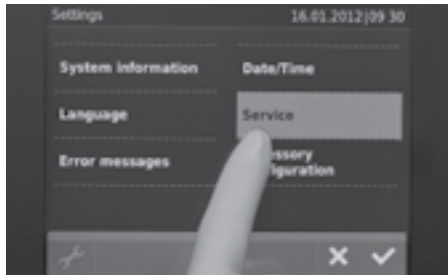
Time settings

Under 'SET TIME' you can perform settings concerning the time format and the current time.

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8.6.5 Service

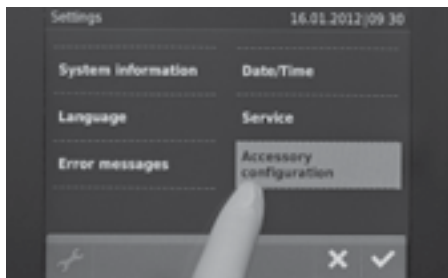
The Service area is password-protected.

i NOTE: Only the KARL STORZ technical customer service or authorized persons are allowed to access the Service area.

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8.6.6 Accessory configuration

In the 'MENU', you can enter the 'ACCESSORY CONFIGURATION' submenu by pressing the 'ACCESSORY CONFIGURATION' button.

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Pedal control

Press 'PEDAL CONTROL' to perform settings for the dual-pedal footswitch.

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Under the 'PUMP' setting, you can set whether the irrigation flow rate is increased or decreased when the left pedal is pressed briefly.

Under 'Shaver <> Sinus Burr' you can enable or disable switching between the shaver and sinus burr when both pedals are pressed at the same time.

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DRILLCUT-X® II

Press 'DRILLCUT-X® II' to perform settings for the DRILLCUT-X® II shaver mode.

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You can select between the Comfort mode or Plus mode for the shaver mode.

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9 Service and repair

9.1 Fuse replacement

Switch off the control unit and disconnect from the mains by pulling out the power plug.

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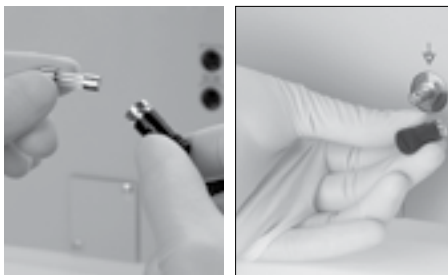
Remove the power fuse holder ⑪ with a screwdriver or another suitable tool.

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! **CAUTION:** Use only fuses with the correct nominal ratings.


Insert new fuses.
Insert the power fuse holder again.

	100-240V AC
Line fuse	2xT4.0 AH 250 V

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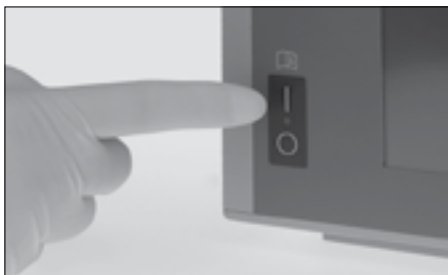Reconnect the power.
Check that the instrument works properly.

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9.2 Cleaning, disinfection, and sterilization

-  **NOTE:** Please read the instruction manual provided with the application instrument carefully!
-  **NOTE:** The ‘Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments’ instructions must be followed. The cleaning, disinfection, and sterilization procedures are explained therein in detail.
-  **NOTE:** For the cleaning/disinfectant solution, use chemicals that have been specially approved by KARL STORZ for the preparation of medical devices. The current lists of chemicals can be found on the Internet at www.karlstorz.com under ‘Hygiene’.

WARNING: Before any cleaning work is performed, the device is to be disconnected from the power source.

CAUTION: Avoid allowing liquids to enter the housing at all costs.

Wipe-clean the exterior surfaces of the device with a disposable, low-lint cloth moistened with disinfectant. For wipe-down disinfection, we recommend using only surface disinfectants based on quaternary ammonium compounds (no solutions predominantly containing alcohol such as rapid-action disinfectants) and ‘disposable disinfectant methods’ based on ammonium chloride.

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9.3 Cleaning, disinfection and sterilization of reusable tubing

WARNING: Reusable tubing must be cleaned, disinfected, and sterilized prior to use and subsequent reuse, paying attention to the manufacturer's specifications.

WARNING: When cleaning and disinfecting contaminated instruments (tubing sets), the local guidelines of the Employer's Liability Insurance Association and equivalent organizations must be observed.

WARNING: Prior to the sterilization of reusable tubing, particularly in cases of manual preparation, it must be ensured that all chemical residue has been rinsed off or removed (see also preparation steps).

WARNING: Disposable tubing sets should not be reused.

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NOTE: The user is responsible for every additional reuse or use of single-use products beyond this point. Disregard results in the exclusion of any liability.

NOTE: The end of the product's service life is determined by wear, preparation methods (chemicals) and damage from use.

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9.3.1 Preliminary cleaning of reusable tubing

Place the reusable tubing in a container filled with a cleaning solution (as per the manufacturer's instructions) immediately after use to prevent contaminants from drying on the surface of the tubing set.

Water quality requirements

Tap water can be used to prepare the cleaning and disinfectant solutions.

When rinsing for the last time, for example, to remove any chemical residue, microbiologically pure/sterile water must be used.

CAUTION: When preparing and using the solutions, follow the manufacturer's instructions paying close attention to proper concentration and exposure times. Prolonged immersion can lead to material changes.

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9.3.2 Preparation

Manual preparation of reusable tubing

WARNING: The local guidelines of the Employer's Liability Insurance Association and equivalent organizations must be observed when performing all cleaning and disinfection procedures on contaminated instruments.

1. Disconnect the reusable tubing from the connectors as necessary and then immerse it in a cleaning solution (made up according to the instructions provided by the manufacturer).
2. Rinse the tubing through under the surface (of the water) using a syringe. Repeat the procedure several times if necessary. Remove soiling with brushes, sponges, and low lint disposable cloths. Cleaning accessories are available from KARL STORZ (see catalog).
3. After cleaning, rinse the tubing and place it in a suitable container with disinfectant solution, making sure that all air bubbles have been removed. Ensure that the tubing is completely covered in the solution for the duration of exposure.
4. Cleaning brushes must be cleaned and disinfected immediately.
5. When the exposure time has passed, remove the tubing and rinse with microbiologically pure/sterile water. Ensure that all chemical residue in and on the tubing is removed.
6. Dry the tubing with sterile compressed air.
7. After preparation, inspect the tubes for residues and damage. If contaminants can still be detected, the preparation procedure must be repeated.

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Machine preparation of reusable tubing

To prepare using a machine, connect the reusable tubing to the retaining mechanisms in such a way that the hollow spaces can be completely rinsed through.

Visible, heavy soiling must be removed by preliminary cleaning prior to machine preparation.

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Assemble tubing set as described in the section ‘Preparing the tubing set’ on page 15, making sure that connections are secure to prevent leakage in the connector area.

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! WARNING: Check the entire tubing set for leaks after preparation. This makes it possible to detect a leak early, through which liquid could escape and enter the device.

To check for leaks use leakage tester 13242 XL, for example. To do so, connect one end to the leakage tester and block off the other end, for example, with your finger. If there is a leak in the tubing set, a pressure drop will be observed on the pressure gauge of the leakage tester.

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9.4 Sterilization

The reusable tubing must be steam sterilized before use at 134°C (+3°C) using the fractionated prevacuum procedure.

WARNING: Successful sterilization is only possible on clean, disinfected surfaces.

WARNING: The recommended sterilization parameters are only valid with sterilization equipment that is properly maintained and validated.

WARNING: Any deviations from the recommended parameters for sterilization must be validated by the user.

9.4.1 Steam sterilization

KARL STORZ has validated the following sterilization method: Fractionated prevacuum method (steam sterilization)

NOTE: Use the fractionated vacuum method (acc. to DIN EN 550/ISO 11134), taking into account the respective applicable national requirements.

The containers are to be placed in the sterilizer so that there is adequate circulation and penetration of steam, air removal and condensate drainage.

Load the sterilizer according to its instruction manual.

At the completion of the steam sterilization cycle, the sterilized instruments must be allowed to cool down slowly.

WARNING: Risk of burns! Parts are hot after steam sterilization. Allow them to cool down.

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**9.5 Fractionated prevacuum
method**

The fractionated prevacuum procedure consists of four phases.

• **Conditioning phase**

A vacuum is created up to four times in the sterilizer. Steam is then blown in.

• **Sterilization phase**

Sterilization takes place at a temperature of 134°C (+3°C) and a (absolute) pressure of 300 kpa (3 bar) over a period of 5 minutes (minimum time).

• **Evacuation phase**

The steam is released.

• **Drying phase**

Drying takes place under renewed application of a vacuum over a period of approx. 5–20 minutes.

Validated parameters for the fractionated prevacuum method:	
Temperature:	134°C (+3°C)
Pressure (p _{absolute}):	300 kpa (3 bar)
Exposure time:	5 minutes for all thermore-sistant medical devices

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9.6 Maintenance and safety check

9.6.1 Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful service life. Maintenance services can be obtained from your local representative or from the manufacturer.

9.6.2 Safety check/repeat inspection according to IEC 62353

WARNING: Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, safety checks/repeat inspections according to IEC 62353 must be carried out on this device and documented at least once a year by qualified personnel.

Visual inspection:

1. Check the device and accessories for any mechanical damage which may impair functionality.
2. Check that inscriptions relevant to safety are legible.

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Electric measurements:

- Inspect the device safety fuses
- Measure protective ground resistance in accordance with IEC 62353: For limit values, please refer to the current norm.
- Measure earth leakage current in accordance with IEC 6235: For limit values, please refer to the current norm.
- Measure touch current in accordance with IEC 6235: For limit values, please refer to the current norm.
- Patient leakage current measured according to IEC 62353: For limit values, please refer to the current norm.

Test for proper functioning:

A functional check must be carried out in accordance with the instruction manual (see § 9.6.2, p. 72).

Documentation

The safety check/repeat inspection and results should be documented.

WARNING: If any defects or shortcomings which could endanger patients, personnel or others are ascertained during these safety checks/repeat inspections, the unit must not be operated until such time as these defects or shortcomings have been eliminated by qualified technical servicing.

NOTE: For detailed information on the range and implementation of the safety check/repeat inspection, please refer to the current Service Manual.

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9.7 Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ genuine KARL STORZ parts only.

9.8 Disposal

9.8.1 Disposal of Equipment

This device has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE).

At the end of its service life, dispose of the device as electronic waste.

Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point.

Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this device.

9.8.2 Disposal of Tubing Sets

No special measures are necessary for disposal. National regulations/laws must be observed.

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9.9 Repair program

For equipment, individual repair is necessary. To bridge the repair period, you usually receive a unit on loan which you then return as soon as you receive the repaired unit.

In the case of repairs, please contact your local KARL STORZ branch or authorized dealer.

Important information

In order to prevent infection, it is strictly forbidden to ship contaminated medical devices. All medical devices must be decontaminated on site, to avoid contact and aerogenous infections (among personnel). We reserve the right to return contaminated instruments or devices to the sender.

Repairs, modifications, or expansions which are not performed by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights. KARL STORZ gives no guarantee on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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9.10 Limitation of liability

KARL STORZ shall be liable for the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- the electrical installation of the location in which the device is connected and operated conforms to the applicable statutes and standards, and
- the device has been used in accordance with its operating instructions at all times.

9.11 Warranty policy

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty specified in our Standard Terms and Conditions of Business shall additionally apply.

The device is always to be sent to your subsidiary (see chapter "Subsidiaries") even during the warranty period.

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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10 Technical description

10.1 General

In the event of the device malfunctioning, the troubleshooting list provided in this chapter will assist the user in finding the possible cause. The troubleshooting list indicates how to find and rectify the various malfunctions.

WARNING: High voltage is present in the device's interior. The device must only be opened by an authorized KARL STORZ Customer Service technician.

WARNING: Maintenance work that is not mentioned in this chapter must only be performed by technical personnel that are authorized by KARL STORZ.

General information

Error messages with the UNIDRIVE® S III ENT are shown directly on the monitor. Hand the device in for servicing when a hardware error message is displayed (E01-E10). Take appropriate action for user error messages (E11-E30). E.g. E11 'NO FOOTSWITCH' – check whether the footswitch is properly connected. Detailed information can be obtained from the error table on page 36 (also see the 'Service mode' section).

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10.2 UNIDRIVE® troubleshooting table

Error message	Cause	Troubleshooting
E01 MEMORY ERROR	Parameters could not be saved successfully	Turn device on/off. Load default values. Or send device in for repair.
E02 MOTORBOARD DEFECTIVE	Motor control overloaded	Turn device on/off. Or send device in for repair.
E03 HARDWARE CONFIGURATION ERROR	Hardware configuration is incorrect	Send device in for repair.
E04 SOFTWARE CONFIGURATION ERROR	Software configuration is incorrect	Send device in for repair.
E05 -		
E06 -		
E07 -		
E08 -		
E09 MOTOR DEFECTIVE M1	Motor on channel 1 is defective	Release footswitch and restart. If the error persists, a different handpiece must be used.
E10 MOTOR DEFECTIVE M2	Motor on channel 2 is defective	See E09
E11 NO FOOTSWITCH	Footswitch not connected correctly	Connect footswitch.
E12 NO MOTOR	No motor or handpiece connected	If the error persists, check whether the error is on the footswitch or device. Send the defective component in for repair.
E13 TOO HIGH TEMPERATURE MOTOR M1	Motor or handpiece on channel 1 overheated	Allow the connected handpiece to cool down or operate an additional handpiece on the 2 nd channel.
E14 -		
E15 TOO HIGH TEMPERATURE MOTOR M2	Motor or handpiece on channel 2 overheated	See E13
E16 IRRIGATION PUMP NOT CLOSED	Pump lever not connected correctly	Set pump lever vertically. If the error persists, the device must be sent in for repair.
E17 MOTOR NOT SUPPORTED M1	The motor or handpiece connected on channel 1 is not supported.	Only use ones approved by KARL STORZ. An update by Service may be necessary.
E18 MOTOR NOT SUPPORTED M2	The motor or handpiece connected on channel 2 is not supported.	See E17
E19 MOTOR STOPPED M1	Motor or handpiece on channel 1 blocked in operation	Release footswitch and restart. If the error persists, clean the handpiece or exchange the attachment.
E20 MOTOR STOPPED M2	Motor or handpiece on channel 2 blocked in operation	See E19
E11 NO SCB MODULE	Device is not connected to an active SCB bus.	Insert SCB cable and activate SCB bus.

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Power switch lamp does not light up.

Possible cause

- 1 Power supply failure.
- 2 Defective power fuse.
- 3 Power switch lamp defective.

Recommended remedy

- 1 Check that there is electricity to the wall outlet.
- 2 Change fuses as described in the instruction manual. Make sure you fit the correct type of fuse.
- 3 Notify customer service.

Problem

- 1 Power switch lamp is illuminated.
- 2 No indication of rotation.
- 3 No energy output when operating the foot-switch/buttons.
- 4 Speed cannot be regulated.

Possible cause

- 1 Faulty motor control electronics.
- 2 Connection to motor interrupted.
- 3a Faulty handpiece.
- 3b Handpiece blocked or jammed.

Recommended remedy

- 1, 2, 4 Notify customer service.
- 3a Connect another handpiece.
- 3b Clean the handpiece.

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Problem

Tubing set connectors do not fit in the pumps.

Possible cause

- 1 Tubing set is assembled incorrectly and upper and lower connectors are the opposite way around.
- 2 Connectors are not attached to the tubing properly.

Recommended remedy

- 1 Reassemble tubing set. (Upper and lower connectors are different sizes.)
- 2 Press connectors firmly in the direction of the pump tubing.

Problem

Irrigation fluid is not flowing.

Possible cause

- 1 Irrigation bag empty.
- 2 Tubing clamp impedes flow.
- 3 Kinked tubing.
- 4 Irrigation bottle not aerated.
- 5 Pump is deactivated.

Recommended remedy

- 1 Use new irrigation bag.
- 2 Loosen tubing clamp.
- 3 Change run of tubing.
- 4 Aerate irrigation bottle.
- 5 Activate pump.

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10.4 Technical data

UNIDRIVE® S III ENT	40701620-1
Line voltage	100-240 V~
Line frequency	50/60 Hz
Line fuse	2 x T 4.0 AH 250 V, IEC 127 format
Power consumption	150-200 W
Operating temperature	10-40°C
Dimensions	305 x 165 x 280 mm (W x H x D)
Weight	5.2 kg
Operating mode (in conjunction with application instrument)	intermittent periodic duty
Relative activation period	50 %; t = 1 min
Operation:	
Humidity:	30 %...70 %
Atmosph. Pressure:	700 ... 1080 hPa
Storage/transport conditions:	
Humidity (non-condensing)	20%...95%
Temperature	(-)10°C...60°C
Atmosph. pressure	500 hPa...1080 hPa
Average lifecycle	5 years
EC micromotor II drill	
Minimum speed:	1,000 rpm
Maximum speed:	40,000 rpm or 80,000 rpm
Start-up maximum speed:	40,000 rpm or 80,000 rpm
Rotation mode:	Rotation
High-Speed motor	
Minimum speed:	1,000 rpm
Maximum speed of the handpieces:	60,000 rpm or 100,000 rpm
Perforator speed	1,200 rpm
Craniotome:	
Minimum speed:	1,000 rpm
Maximum speed:	60,000 rpm

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DRILLCUT-X® II – Shaver

Minimum speed: 300 rpm
Maximum speed: 10,000 rpm
Start-up maximum speed: 4,000 rpm
Rotation mode: Oscillation

DRILLCUT-X® II – Sinus drill

Minimum speed: 300 rpm
Maximum speed: 12,000 rpm
Start-up maximum speed: 12,000 rpm
Rotation mode: Rotation

DRILLCUT-X® – Shaver

Minimum speed: 300 rpm
Maximum speed: 7,000 rpm
Start-up maximum speed: 7,000 rpm
Rotation mode: Oscillation

DRILLCUT-X® – Sinus drill

Minimum speed: 800 rpm
Maximum speed: 12,000 rpm
Start-up maximum speed: 12,000 rpm
Rotation mode: Rotation

INTRANASAL DRILL

Minimum speed: 3,000 rpm
Maximum speed: 60,000 rpm
Start-up maximum speed: 60,000 rpm
Rotation mode: Rotation

SAWS

Minimum speed: 1,000 rpm
Maximum speed 1: 20,000 rpm
Maximum speed 2: 15,000 rpm
Start-up maximum speed 1: 20,000 rpm
Start-up maximum speed 2: 15,000 rpm
Rotation mode: Rotation

DERMATOMES

Minimum speed: 1,000 rpm
Maximum speed: 8,000 rpm
Start-up maximum speed: 8,000 rpm
Rotation mode: Rotation

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10.5 Standard compliance
(for 40701420-1)

According to IEC 60601-1, UL 60601-1, CAN/CSA C22.2 No. 601.1-M90:

- Type of protection against electric shock: Protection Class I
- Degree of protection against electric shock: Applied part type BF
- Type of protection against moisture: Drip-water protection as per IPX 1

Conforms with IEC 60601-1-2 :2001:

Observe the information on electromagnetic compatibility in the appendix (pages 97-109).

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10.6 Directive compliance

Medical product of class IIa

This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC.

NOTE: The code number after the CE mark indicates the responsible notified body.

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10.7 Technical documentation

On request, the manufacturer will provide available circuit diagrams, detailed parts lists, descriptions, instructions for adjustments, and other documentation deemed useful for suitably qualified user personnel authorized by the manufacturer when repairing device components that have been designated as repairable by the manufacturer.

Supply of such technical documentation relating to the device shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the device.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

i **NOTE:** We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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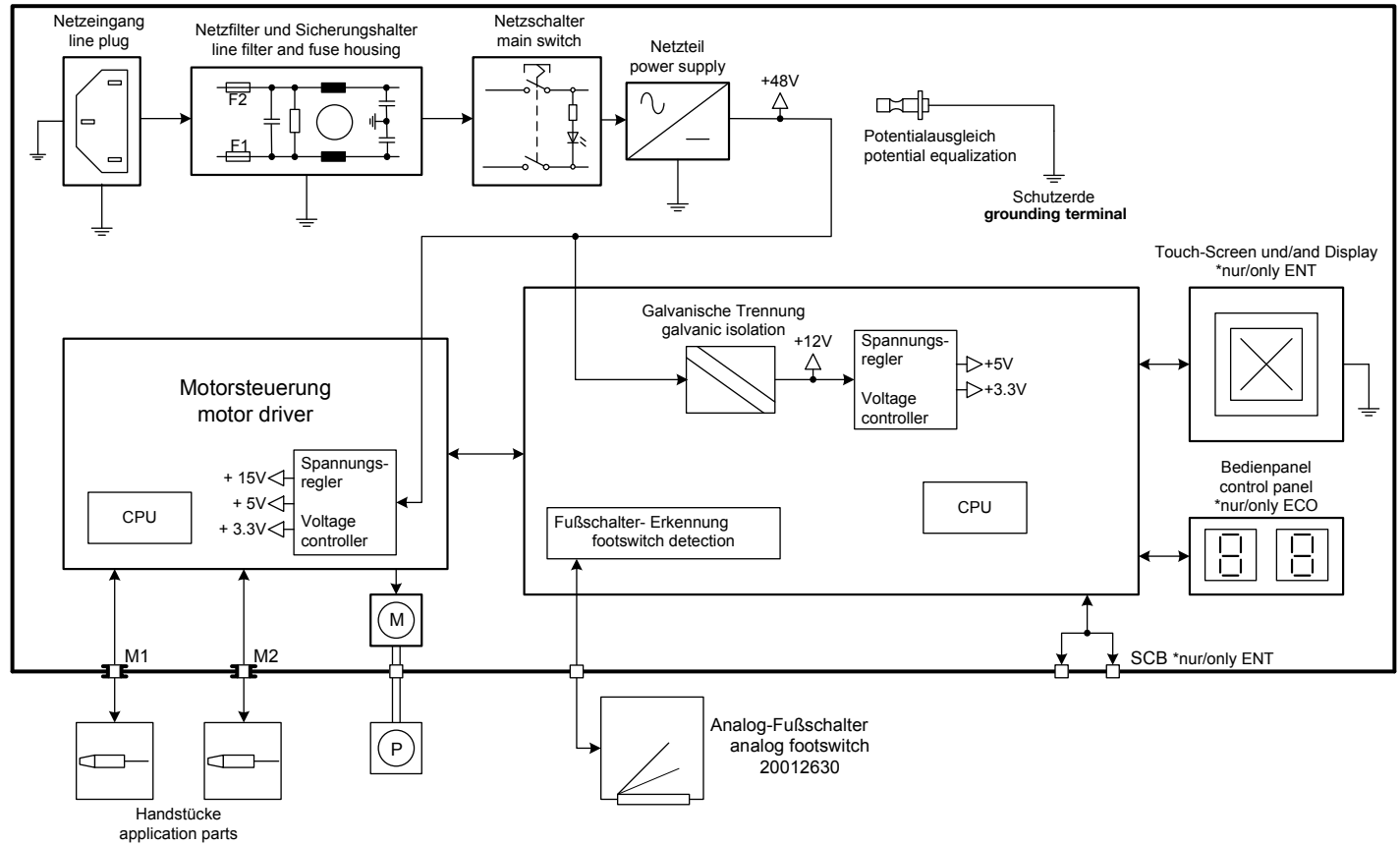
10.8 Block diagram

10.8 Diagrama funcional

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**11 Spare parts,
recommended accessories**

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Part	Order no.
Dual-pedal footswitch, two-stage, with proportional function	20016630
Dual-pedal footswitch, two-stage, with proportional function, 6 m connecting cord	20016631
Single-pedal footswitch, single-stage, with proportional function	20016230
Silicone tubing set, for irrigation, reuseable, sterilizable	20711640
Clip set, for tubing set	20711621
Pump tubing, package of 25, sterilizable	20330393
Line fuse T 4.0 AH 100-240V AC	2027690
SCB connecting cable, length 100 cm	20090170
Irrigator rod	20711680
Mains cord (grounded)	400 A
'Hospital Grade' mains cord (USA)	400 B
Instruction manual	96056025D

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Part	Order no.
Tubing set, for single use, for use with UNIDRIVE® S III ENT, sterile, package of 10	mtp*
Universal spray, pack of 6 cans	280053

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**Recommended accessories in combina-
tion with high-performance EC micromotor
20711033 and connecting cord 20711173**

(Not included in the scope of supply)

Part	Order no.
INTRA drill handpiece, angled, 18 cm, corresponding drills: standard, diamond, coarse diamond	252572
Rack for 12.5 cm drills, folding model	280034
INTRA drill handpiece, angled, 15 cm, corresponding drills: standard, diamond, coarse diamond	252591
Rack for 9.5 cm drills, folding model	280033
INTRA drill handpiece, angled, 12.5 cm *	252570
INTRA drill handpiece, straight, long, 10.4 cm *	252591
INTRA drill handpiece, straight, 8.7 cm *	252590
* Corresponding drills: standard, tungsten carbide, tungsten carbide cross cut, diamond, coarse diamond	
Tray holder for drill/burr system without rack	39552 A
Tray holder for drill/burr system with rack	39552 B
Micro sagittal saw	254100
Osseoscalpel	254200
Micro compass saw	254300
Oscillating micro saw	254000
Tray holder for micro saw system	39553 A

* mtp medical technical promotion gmbh,
Take-Off GewerbePark 46
78579 Neuhausen ob Eck, Germany
Phone: +49 (0)7467 945 04-0
Fax: +49 (0)7467 945 04-99

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**Recommended accessories in combination
with high-performance EC micromotor II
20711033 and connecting cord 20711173**

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**Recommended accessories in combination
with the High-Speed micro motor 20712033**

Part	Order no.
High-Speed drill handpiece, extra long, slim, angled, 60,000 rpm for use with the High-Speed micromotor 20712033	252671
High-Speed drill handpiece, super long, slim, angled, 60,000 rpm for use with the High-Speed micromotor 20712033	252672
High-Speed drill handpiece, extra short, angled, 60,000 rpm for use with the High-Speed micromotor 20712033	25266x
High-Speed drill handpiece, short, angled, 100,000 rpm for use with the High-Speed micromotor 20712033	25268x
High-Speed drill handpiece, extra short, straight, 60,000 rpm for use with the High-Speed micromotor 20712033	25269x

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**Recommended accessories in combination
with DRILLCUT-X® 40711050, DRILLCUT-X® II
40712050 and DRILLCUT-X® II N 40712055**

(Not included in the scope of supply)

Shaver blades: straight, 35° curved, 65° curved non-serrated, single- serrated, double-serrated, reusable or disposable (ster- ile), pack of 5.	Article number, see catalog
Sinus burr: oblong, round drill, diamond head 15° curved, 40° curved, 55° curved, disposable (sterile)	Article number, see catalog

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12 Appendix**12.1 Cleaning agents and disinfectants****Chemicals for the preparation of telescopes and endoscopic instrument sets****12.1.1 Manual preparation**

The following instruments and telescopes produced by KARL STORZ must not be fully immersed in a liquid: HAMOU® telescopes with ocular focusing, magnifiers with focusing, motors, and their handpieces, IMPERATOR drill handpieces, prismatic light deflectors. The following instruments produced by KARL STORZ are not suitable for cleaning in an ultrasound cleaning bath: rigid telescopes, flexible telescopes, other instruments with optical components made of glass. When preparing and using the solutions, follow the solution manufacturer's instructions paying close attention to proper dilution and recommended immersion times. Regarding the microbiological effectiveness, please ask the manufacturer.

The materials approved by KARL STORZ as regards material compatibility can be found on the following page.

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Type

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths
- A = Not suitable for flexible endoscopes
- (A) = Only for new generation flexible endoscopes (year of manufacture >2005)

i NOTE: You will also find the current list of approved materials on the Internet at www.karlstorz.com.

i NOTE: When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

! * CAUTION: The use of CAUSTIC SODA can cause changes to the surface of aluminum parts (including coated parts), plastics, and soldered connections, and may reduce the service life of the instrument set.

For flexible endoscopes, only immerse the examination shaft, not the housing/handle part.

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12.1.2 Machine preparation

The following instruments and telescopes produced by KARL STORZ must not be prepared in a cleaning and disinfecting machine: HAMOU® telescopes with ocular focusing, magnifiers with focusing, motors, and their handpieces, IMPERATOR drill handpieces, standard, tungsten carbide and diamond-tipped drills, prismatic light deflectors, and magnetic instrument racks.

Selection of a method for cleaning and disinfection must be discussed with the machine manufacturer and the manufacturer of the chemical reagents. Only special methods must be used which have been verified for this purpose. Regarding the microbiological effectiveness, please ask the manufacturer. Thermal disinfection is preferable.

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12.2 Electromagnetic Compatibility (EMC) information

WARNING: Medical electrical equipment needs special precautions regarding electromagnetic compatibility (EMC). Observe the EMC instructions in this appendix during installation and commissioning.

The UNIDRIVE® S III ENT model 40701620-1 corresponds to EN/IEC 60601-1-2:2001 [CISPR 11 Class B] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The UNIDRIVE® S III ENT model 40701620-1 is a Group 1 device (as per CISPR 11).

Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

i NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the device or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the device or system to perform its intended use without causing interferences in other devices or systems or non-medical electrical devices. If this device does cause electromagnetic interferences with other devices, the user is encouraged to try to correct the interferences by one or more of the following measures:

- reorient or relocate the receiving device;
- increase the distance between the equipment;
- connect the devices to different electrical circuits.

If you have any further questions, please contact your local representative or our service department.

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 **WARNING:** The UNIDRIVE® S III ENT model 40701620-1 should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device or system should be observed to verify normal operation in the configuration in which it will be used.

 **WARNING:** The use of portable and mobile RF equipment may have an impact on this or other pieces of medical equipment.

 **WARNING:** The use of accessories and cables other than those specified in the KARL STORZ instruction manual may result in increased emissions or decreased immunity of the UNIDRIVE® S III ENT model 40701620-1. The accessories and cables listed below have been shown to comply with the requirements of EN/IEC 60601-1-2. When using accessories and cables other than those specified here, it is the responsibility of the user to ensure that they comply with EN/IEC 60601-1-2.

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Table 201 Guidelines and manufacturer's declaration – electromagnetic emissions		
The UNIDRIVE® S III ENT model 40701620-1 is intended for use in the electromagnetic environment specified below. The user of the UNIDRIVE® S III ENT model 40701620-1 should ensure that it is used in such an environment.		
Interference immunity test	Conformity level	Electromagnetic environment – guidance
RF emissions acc. to CISPR 11	Group 1	The UNIDRIVE® S III ENT model 40701620-1 uses RF energy solely for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions acc. to CISPR 11	Class B	The UNIDRIVE® S III ENT model 40701620-1 is suitable for use in all establishments (e.g., hospitals and doctors' offices) including domestic environments and those directly connected to the public power supply network that also supplies buildings used for domestic purposes.
Emission of harmonic oscillations acc. to IEC 61000-3-2	Class D (see NOTE below)	
Voltage fluctuations/flicker emissions acc. IEC 61000-3-3	not applicable (see NOTE below)	
<p>i NOTE: Measurement of harmonic oscillations not necessary (output < 75 W) Measurement of voltage fluctuations/flicker not necessary (manually activated, occurrence of significant voltage fluctuations/flicker unlikely)</p>		

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Table 202			
Guidelines and manufacturer's declaration – electromagnetic immunity			
The UNIDRIVE® S III ENT model 40701620-1 is intended for use in the electromagnetic environment specified below. The user of the UNIDRIVE® S III ENT model 40701620-1 should ensure that it is used in such an environment.			
Interference immunity tests	EN/IEC 60601 test level	Conformity level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV Contact discharge ± 8 kV Air discharge	6 kV Contact discharge 8 kV Air discharge	Floors should be made of wood or concrete or provided with ceramic tiles. If floors are covered with synthetic material, the relative humidity must be at least 30%.
Rapid transient electric interferences/bursts acc. to IEC 61000-4-4	± 2 kV for power lines ± 1 kV for input and output lines	2 kV/1 kV	The quality of the supply voltage should be that of a typical business or hospital environment.
Surges acc. to IEC 61000-4-5	± 1 kV Differential mode ± 2 kV Common mode	2 kV/1 kV	The quality of the supply voltage should be that of a typical business or hospital environment.
Voltage dips, short interruptions and fluctuations of the supply voltage acc. to IEC 61000-4-11	<5% U_T * (>95% dip of the U_T) for ½ cycle	0 V, 10 ms	The quality of the supply voltage should be that of a typical business or hospital environment. If the user of the device requests continued functioning even when interruptions of the power supply occur, it is recommended to supply the device from a power supply that is free of interruptions.
	40% U_T (60 % dip of the U_T) for 5 cycles	92 V, 100 ms	
	70% U_T (30% dip of the U_T) for 25 cycles	168 V, 500 ms	
	<5% U_T (>95% dip of the U_T) for 5 seconds	0 V, 5 sec	
Magnetic field at the supply frequency (50/60 Hz) acc. to IEC 61000-4-8	3 A/m	3 A/m	The magnetic fields at the mains frequency should be at the values typical of business or hospital environments.

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Table 204			
Guidance and manufacturer's declaration – electromagnetic immunity – for equipment and systems that are not life-supporting			
The UNIDRIVE® S III ENT model 40701620-1 is intended for use in the electromagnetic environment specified below. The user of the device should make sure that it is used in such an environment.			
Interference immunity tests	EN/IEC 60601 test level	Conformity level	Electromagnetic environment – guidance
<p>Conducted RF disturbances acc. to IEC 61000-4-6</p> <p>Radiated RF disturbance acc. to IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to < 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 Vrms</p> <p>3 V/m</p>	<p>Portable and mobile radio equipment, including cables, should not be used at a shorter distance from the UNIDRIVE® S III ENT model 40701620-1/40701720-1/40701820-1 than the recommended safety distance, which is calculated by the equation relevant for the transmitting frequency.</p> <p>Recommended separation distance:</p> $d = [3.5/3]\sqrt{P}$ <p>150 kHz to < 80 MHz</p> $d = [3.5/3]\sqrt{P}$ <p>80 MHz to < 800 MHz</p> $d = [7/3]\sqrt{P}$ <p>800 MHz to 2.5 GHz</p> <p>Where P is the nominal power of the transmitter in watts [W] according to the information provided by the manufacturer of the transmitter and d is the recommended separation distance in meters [m].</p> <p>The field strength of stationary transmitters at all frequencies on site ^a should be, according to a study, less than the conformity level. ^b</p> <p>Interferences may occur in the vicinity of devices that carry the following symbol:</p> 
<p>Note: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of buildings, objects, and persons.</p>			
<p>^a Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location at which the device is used exceeds the above conformity levels, the device should be observed to verify normal operation. If unusual performance characteristics are observed, additional measures may be necessary, such as a modified alignment or a different location for the device.</p> <p>^b Over the frequency range from 150 kHz to 80 MHz, the field strength should be less than 3 V/m.</p>			

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Table 206

Recommended separation distances between portable and mobile HF communications equipment and the UNIDRIVE® S III ENT model 40701620-1

The UNIDRIVE® S III ENT model 40701620-1 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the device can help avoiding electromagnetic interferences by observing the minimum distance between portable and mobile RF telecommunication devices (transmitters) and the device – as a function of the output power of the communication device, as shown below.

Nominal power of transmitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to < 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to < 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters whose maximum nominal power is not listed in the above table, the recommended separation distance d in meters (m) can be determined using the equation belonging to the relevant column, where P is the maximum nominal power of the transmitter in Watts (W) as specified by the manufacturer of the transmitter.

Note: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of buildings, objects and persons.

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KARL STORZ Endoscopy Canada Ltd.
7171 Millcreek Drive, Mississauga
Ontario, L5N 3R3, Canada
Phone: +1 905 816-4500, Fax: +1 905 858-4599
E-Mail: info.canada@karlstorz.com

KARL STORZ Endoscopy-America, Inc.
2151 East Grand Avenue
El Segundo, CA 90245-5017, USA
Phone: +1 424 218-8100
Toll free: 800 421-0837 (USA only)
Fax: +1 424 218-8526
Toll free: 800 321-1304 (USA only)
E-Mail: info@ksea.com

KARL STORZ Veterinary Endoscopy America, Inc.
175 Cremona Drive
Goleta, CA 93117, USA
Phone: +1 805 968-7776, Fax: +1 805 685-2588
E-Mail: info@karlstorzvet.com

KARL STORZ Endoscopia Latino-America, Inc.
815 N. W. 57th Avenue, Suite 480
Miami, FL 33126-2042, USA
Phone: +1 305 262-8980, Fax: +1 305 262-8986
E-Mail: info@ksela.com

KARL STORZ Endoscopia
Miramar Trade Center
Edificio Jerusalem, Oficina 108,
La Habana, Cuba
Phone: +53 72041097, Fax: +53 72041098

KARL STORZ Endoscopia México S.A. de C.V
Lago Constanza No 326, Col. Chapultepec Morales
D.F.C.P. 11520, México, México
Phone: +52 55 1101-1520, Fax: +52 55 55450174

KARL STORZ Marketing América Do Sul Ltda.
Rua Joaquim Floriano, nº. 413, 20º andar – Itaim Bibi,
CEP-04534-011 São Paulo, Brasil
Phone: +55 11 3526-4600, Fax: +55 11 3526-4680
E-Mail: info@karlstorz.com.br

KARL STORZ Endoscopia Argentina S.A.
Zufriategui 627 6º Piso
B1638 CAA - Vicente Lopez
Provincia de Buenos Aires, Argentina
Phone: +54 11 4718 0919, Fax: +54 11 4718 2773
E-Mail: info@karlstorz.com.ar

KARL STORZ Endoskopi Norge AS
Østsjøveien 15B
0661 Oslo, Norway
Phone: +47 6380 5600, Fax: +47 6380 5601
post@karlstorz.no

KARL STORZ Endoskop Sverige AB
Storsåtragränd 14,
12724 Skårholmen, Sweden
Postal address: Po Box 8013,
14108 Kungens Kurva, Sweden
Phone: +46 8 50 56 4800, Fax: +46 8 50 56 4848
E-Mail: kundservice@karlstorz.se

KARL STORZ Endoscopy Suomi OY
Valimotie 1 B, 4. kerros, 00380 Helsinki, Finland
Phone: +358 (0)96824774, Fax: +358 (0)968247755
E-Mail: asiakaspalvelu@karlstorz.fi

KARL STORZ GmbH & Co. KG
Representation Office
Kestucio g. 59
08124 Vilnius, Lithuania

KARL STORZ Endoskopi Danmark A/S
Skovlytoften 33
2840 Holte, Denmark
Phone: +45 45162600, Fax: +45 45162609
E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy (UK) Ltd.
392 Edinburgh Avenue, Slough
Berkshire, SL1 4UF, Great Britain
Phone: +44 1753 503500, Fax: +44 1753 578124
E-Mail: customerservice@karlstorz-uk.com

KARL STORZ Endoscopia Nederland B. V.
Phone: +31 651 938 738, Fax: +31 135 302 231

KARL STORZ Endoscopy Belgium N. V.
Phone: +32 473 810 451,
E-Mail: info@stopleer.be

KARL STORZ Endoscopia France S. A. S.
12, rue Georges Guynemer, Quartier de l'Europe
78280 Guyancourt, France
Phone: +33 1 30484200, Fax: +33 1 30484201
E-Mail: marketing@karlstorz.fr

KARL STORZ Endoskop Austria GmbH
Landstraße Hauptstr. 148/1/G1
1030 Wien, Austria
Phone: +43 1 71 56 0470, Fax: +43 1 71 56 0479
E-Mail: storz-austria@karlstorz.at

KARL STORZ Endoscopia Ibérica S. A.
Parque Empresarial San Fernando
Edificio Munich – Planta Baja
28830 Madrid, Spain
Phone: +34 91 6771051, Fax: +34 91 6772981
E-Mail: marketing@karlstorz.es

KARL STORZ Endoscopia Italia S. r. l.
Via dell'Artigianato, 3
37135 Verona, Italy
Phone: +39 045 8222000, Fax: +39 045 8222001
E-Mail: info@karlstorz.it

KARL STORZ Adria Eos d.o.o.
Zadarska 80
10000 Zagreb, Croatia
Phone: +385 1 6406 070, Fax: +385 1 6406 077
E-Mail: info@karlstorz.hr

KARL STORZ Endoskopija d.o.o.
Verovškova c. 60A
1000 Ljubljana, Slovenia
Phone: +386 1 620 5880, Fax: +386 1 620 5882
E-Mail: pisarna@karlstorz.si

KARL STORZ Endoscopia Romania srl
Str. Prof. Dr. Anton Colorian, nr. 74, Sector 4
041393 Bukarest, Romania
Phone: +40 (0)31 4250800, Fax: +40 (0)31 4250801
E-Mail: office@karlstorz.ro

KARL STORZ Endoskope Greece M.E.P.E.
Gounari 40,
15343 Agia Paraskevi/Athens, Greece
Phone: +30 210 61 31 386, Fax: +30 210 61 31 392
E-Mail: sales@karlstorz.gr

KARL STORZ Endoskope Greece Ltd.*
Ipsilantou Str. 32
54248 Thessaloniki, Greece
Phone: +30 2310 304868, Fax: +30 2310 304862
*Repair & Service Subsidiary

KARL STORZ Industrial**
Gedik Is Merkezi B Blok
Kat 5, D 38-39, Bagdat Cad. No: 162
Maltepe Istanbul, Turkey
Phone: +90 216 442 9500, Fax: +90 216 442 9030
**Sales for Industrial Endoscopy

000 KARL STORZ Endoscopy – WOSTOK
Derbenyevskaya nab. 7, building 4
115114 Moscow, Russia
Phone: +7 495 983 02 40, Fax: +7 495 983 02 41
E-Mail: kste-wostok@karlstorz.ru

TOV KARL STORZ Ukraine
Obolonska naberezhna, 15
building 3, office 3
04210 Kiev, Ukraine
Phone: +380 44 42668-14, -15, -19
Fax: +380 44 42668-20

KARL STORZ Endoskope
East Mediterranean and Gulf S.A.L.
Block M, 3rd Floor
Beirut Souks, Weygand Street
2012 3301 Beirut, Lebanon
Phone: +961 1 1999390, Fax: +961 1 1999391
E-Mail: info@karlstorz-emg.com

KARL STORZ Endoscopy (South Africa) (Pty) Ltd.
P.O. 6061, Roggebaai 8012, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

KARL STORZ Endoscopy Kazakhstan LLP
6, Saryarka str. BC „Arman“, office430
010000 Astana, Republic of Kazakhstan
Phone: +7 7172 552549, 552788,
Fax: +7 7172 552444
E-Mail: info@karlstorz.kz

KARL STORZ ENDOSKOPE East Mediterranean & Gulf
Building 25 (Al-Hayyan) – Units 205 – 206
Dubai Health Care City, P.O.Box 118069
Dubai - United Arab Emirates
Phone: +971 (0)4 4471230,
Fax: +971 (0)4 4471231
E-Mail: info-gne@karlstorz-emg.com

KARL STORZ Endoscopy India Private Ltd.
D-181, Okhla Industrial Area, Phase-1
New Delhi 110020, India
Phone: +91 11 26815445-51,
Fax: +91 11 268129 86
E-Mail: karlstorz@vsnl.com

KARL STORZ GmbH & Co. KG
Resident Representative Office
80/33 (44/19) Dang Van Ngu,
F.10-Q. Phu Nhuan
Ho Chi Minh City, Vietnam
Phone: +848 991 8442, Fax: +848 844 0320

KARL STORZ Endoscopy Australia Pty Ltd
15 Orion Road Lane Cove NSW 2066
P O Box 50 Lane Cove NSW 1595, Australia
Phone: +61 (0)2 9490 6700
Toll free: 1800 996 562 (Australia only)
Fax: +61 (0)2 9420 0695
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd.
Hong Kong Representative Office
Unit 1601, Chinachem Exchange Square
1 Hoi Wan Street, Quarry Bay,
Hong Kong, People's Republic of China
Phone: +8 52 28 65 2411, Fax: +8 52 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy (Shanghai) Ltd.
Beijing Branch Company,
Room 610, China Life Tower No. 6, Chaowai Street,
Beijing, 100020, People's Republic of China
Phone: +86 10 8525 3725,
Fax: +86 10 8525 3728
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Unit 3901-3904, Tower 1 Grand Gateway
No. 1 Hong Qiao Road
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188, Fax: +86 21 6113-1199
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Chengdu Branch Company
F-5, 24/F., Chuanxing Mansion,
No. 18 Renming Road South
Chengdu, Sichuan, 610016, People's Republic of China
Phone: +86 28 8620-0175, Fax: +86 28 8620-0177
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Shenyang Branch Company,
Rm 2225, Tower B, City Plaza,
No. 83, Zhongshan Road, Heping District,
Shenyang, Liaoning, 110001,
People's Republic of China
Phone: +86 24 6258 9911, Fax: +86 24 6258 9922
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Guangzhou Branch Company
Room 1119-20, Dongshan Plaza
69 Xianlie Road Middle, Dongshan District,
Guangzhou, Guangdong, 510095,
People's Republic of China
Phone: +86 20 8732-1281, Fax: +86 20 8732-1286
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy Asia Marketing Pte Ltd
No. 8 Commonweath Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com.sg

KARL STORZ Endoscopy Singapore Sales Pte Ltd
No. 8 Commonweath Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com.sg

KARL STORZ Endoscopy Korea Co. Ltd.
Jelton Tower 1, Suite 905
16 Neuti-Ro, Bundang-Gu
Seongnam, 463-847, Korea
Phone: +82-70-4350-7474, Fax: +82-70-8277-3299
E-Mail: stephen.Lee@karlstorz.com

KARL STORZ Endoscopy Taiwan Ltd.
6F-1, No. 10, Sec. 1, Beisin Rd.
Sindian, Taipei County 231, Taiwan (R.O.C.)
Phone: +886 933 014 160, Fax: +886 2 8672 6399
E-Mail: mingwang@karlstorz.tw

KARL STORZ Endoscopy Japan K. K.
Stage Bldg. 8F, 2-7-2 Fujimi
Chiyoda-ku, Tokyo 102-0071, Japan
Phone: +81 3 6380-8622, Fax: +81 3 6380-8633
E-Mail: info@karlstorz.co.jp
www.karlstorz.com

 **KARL STORZ GmbH & Co. KG**

Mittelstrasse 8
78532 Tuttlingen

Postfach 230
78503 Tuttlingen
Germany

Telefon: +49 (0)7461 708-0
Telefax: +49 (0)7461 708-105
E-Mail: info@karlstorz.de
Web: www.karlstorz.com