

...Tilt table with fixed height and lying surface tiltable

Please choose - without surcharge – between 2 different variants of leatherettes:

- Stamskin®Top (34 colours) or - Skai® Pandora Plus (20 colours)



Model 90-00: Frame white powder coated

Model 90-04: Frame silver powder coated

Tilt table with fixed height (no height adjustment) and Tilt-Function by electric motor adjustment.

Basic equipment:

Lying surface 1-sectioned (without adjustable head part (= option), different possibilities see schedule below)

- Foot plate (to stand on) removable.

For usage the foot plate is stucked in 2 holders and fixed.

- Lying surface **tiltable by electric motor from -15° to +85°**.

The motor is controlled by hand and foot switch

Time of electric tilt adjustment from min. to max.: 22 sec.

(Faster movement for tilting is possible as option, but then loss of motor power. For more informations please contact us.)

- Movable on 4 double-wheels castors, 75 mm diam., each with total lock

- Special side rails along lying surface (left and right side)

for special tilt table equipment. (Please see schedule below)

- Thickness of upholstery: 6 cm. Upholstery base: Laminated wood

Measures lying surface: Length: 1.930 mm, Width: 700 mm


Height (lying surface in horizontal position): 700 mm

Load capacity for max. patient weight:

180 kg (central burdened; patient in lying position)

Measures overall: Length: 1.960 mm, Width: 850 mm

Necessary room height: 2,15 m (without extension rail)

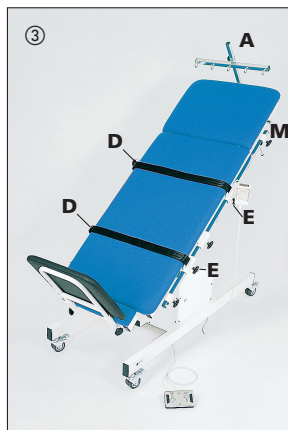

Special equipment for Model 90-00 / 90-04	Representation: Page, Picture-No.	Code	Mod.-No.
Extension rail	Page 30, Picture 3	A	90-025
1 Pair of little grip handles	Page 30, Picture 1	B	90-028
Upholstery for strap (for pressure distribution)	Page 30, Picture 5	C	90-034
1 Safety belt	Page 30, Pictures 1, 3, 5	D	90-035
1 Pair attachment clamps (Necessary to fix the belt at the side rails of the tilt table)	Page 30, Pictures 1, 3, 5	E	90-036
Work and rest desk, up-/downwards adjustable	Page 30, Picture 1	F	90-037
Work and rest desk, rigid	Page 30, Picture 2	G	90-039
1 Pair of big grip handles	Page 31, Pictures 1, 2	H	90-038
Head part up-/downwards adjustable by gas spring. (then without holder for extension rail)	Page 30, Pictures 4, 5	K	90-041
Foot plate (to stand on) up-/downwards adjustable by electric motor (then not removable)	Page 30, Pictures 4, 5	L	90-043
Head section upwards manual adjustable	Page 30 and 31, Pictures 1-3	M	90-058
Angle gauge (bothsides the table)	without picture	-	90-090
Width of lying surface 800 mm	without picture	-	90-080
Arm rest	Page 30, Picture 4	N	90-059
Attachment clamp to fix arm rest at side rail	Page 30, Picture 4	O	90-143
Paper roll holder	without picture	-	90-1272

For information:

The special equipment of Tilt tables Mod. 90 and 2090 is nearly the same, excepted head adjustability by gas spring.

Please notice: All pictures 1 – 5 showing Model 90-00 with special equipment. The special equipment is marked in the pictures with letters. Please use the schedule to decode and identify it to provide the table with special equipment for your special applications.


Tables with electric motor powered adjustment must have a safety-lock-box to protect against unauthorized usage (regulations of BfArM based on EU-ordinance acc. EN 60601). Please choose between our 2 possibilities:

- Mod. 1993: Safety box, fixed on the underpart, with 1 magnetic key (see picture on page 34)

- Mod. 1995: Emergency stop button, fixed on top frame, with 2 keys (see picture on page 34)

...Tilt table with

- height adjustment by electric motor
- lying surface tiltable by electric motor

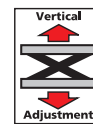

Please choose - without surcharge – between 2 different variants of leatherettes:
- Stamskin®Top (34 colours) or - Skai® Pandora Plus (20 colours)



Model 2090-00: Frame white powder coated

Model 2090-04: Frame silver powder coated

With electric motor drive for height and tilting adjustment of lying surface
Particularity: Complete tilt function possible in each height of the table.

Complete height adjustment (max. or min.) possible in each surface level position. (Both possibilities of adjustment are independent of each other)

Basic equipment:

Lying surface 1-sectioned (without adjustable head part (= option), different possibilities see schedule below.)

- Foot plate (to stand on) removable.

- For usage the foot plate is stuck in 2 holders and fixed.

- With 2 independent working electric motors:

One for height adjustment, one for tilting.

- Complete lying surface **tiltable by electric motor from 0° to +85°** controlled by hand switch.

Time of electric **tilt adjustment min. to max.: 22 sec.**

(Faster movement for tilting is possible as option, but then loss of motor power. For more informations, please contact us.)

- **Height adjustment** by electric motor: **590 mm to 900 mm** controlled by foot switch. Time of height adjustment min. to max.: 34 sec.

- Movable on 4 double-wheels castors, 75 mm diam., each with total lock

- Special side rails along lying surface (left and right side) for special tilt table equipment. (Please see schedule below)

- Thickness of upholstery: 6 cm. Upholstery base: Laminated wood

Measures lying surface: Length: 1.930 mm, Width: 700 mm

Height adjustable from 590 mm to 900 mm

Load capacity for max. patient weight:

180 kg (central burdened; patient in lying position)

Measures overall: Length: 1.960 mm, Width: 850 mm

Necessary room height: 2,40 m (without extension rail)



Please notice: All pictures 1 – 3 showing Model 2090-00 with special equipment. The special equipment is marked in the pictures with letters. Please use the schedule to decode and identify it to provide the table with special equipment for your special applications.

Please note:
Also special equipment for Model 2095

Special equipment for Model 2090-00 / 2090-04	Representation: Page, Picture-No.	Code	Mod.-No.
Extension rail	Page 30, Picture 3	A	2090-025
1 Pair of little grip handles	Page 30, Picture 1	B	2090-028
Upholstery for strap (for pressure distribution)	Page 30, Picture 5	C	2090-034
1 Safety belt	Page 30, Pictures 1, 3, 5	D	2090-035
1 Pair attachment clamps (Necessary to fix the belt at the side rails of the tilt table)	Page 30, Pictures 1, 3, 5	E	2090-036
Work and rest desk, up-/downwards adjustable	Page 30, Picture 1	F	2090-037
Work and rest desk, rigid	Page 30, Picture 2	G	2090-039
1 Pair of big grip handles	Page 31, Pictures 1, 2	H	2090-038
Head part upwards adjustable by gas spring. (then without holder for extension rail)	without picture	-	2090-042
Foot plate (to stand on) up-/downwards adjustable by electric motor (then not removable)	Page 30, Pictures 4, 5	L	2090-043
Head section upwards manual adjustable	Page 30 and 31, Pictures 1-3	M	2090-058
Angle gauge (bothsides the table)	without picture	-	2090-090
Width of lying surface 800 mm	without picture	-	2090-080
Arm rest	Page 30, Picture 4	N	2090-059
Attachment clamp to fix arm rest at side rail	Page 30, Picture 4	O	2090-143
Paper roll holder	without picture	-	2090-1272

For information:

The special equipment of Tilt tables Mod. 90 and 2090 is nearly the same, excepted head adjustability by gas spring.

Tables with electric motor powered adjustment must have a safety-lock-box to protect against unauthorized usage (regulations of BfArM based on EU-ordinance acc. EN 60601). Please choose between our 2 possibilities:

- Mod. 1993: Safety box, fixed on the underpart, with 1 magnetic key (see picture on page 34)

- Mod. 1995: Emergency stop button, fixed on top frame, with 2 keys (see picture on page 34)