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GREINER work stools and chairs are available in many different versions: with hand or foot release of the height adjustment, different seat cushions, backrests and seat heights. For further information please refer to our **workstools** brochure.

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careLine **hv** with manually adjustable backrest and legrest and extendable leg cushion, head bolster as accessory. p/n 4460221

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careLine **fix** with manually adjustable backrest and legrest and extendable leg cushion, head bolster as accessory. p/n 4460121

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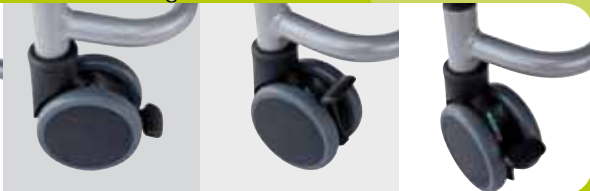
front double castors with 3 lever positions: free to move, directional stability, totally blockable

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adjustment of the backrest and legrest manually / electrical

Zubehör
accessories

Seiten
pages

10-11
10-11

technical information

chassis

fix: The chair is not adjustable in height, seat height is 51-58 cm (depending on the adjustment of the seat angle).

hvh: Hydraulically adjustable in height from 51-78 cm (depending on the adjustment of the seat angle) by a lever **(1)** mounted on both sides.

castors: By means of 4 double castors **(2)**, the chair can easily and precisely be moved in all directions. The rolling direction adjustment provides straight running stability – the 4 brakes keep the chair firmly stationary when applied.

adjustment of the backrest and legrest

manually: The backrest and legrest are separately adjustable to the horizontal by a gas spring. **(3)**

electrically: The backrest and legrest can be adjusted separately by a remote control **(3)**. The Lounger is equipped with a back up battery to enable temporary operation independent of mains electric supply.

adjustment of the upper part **(4)**

The possibility to incline the seat (after having loosened the arrester) allows a standing up aid and reduces therefore the effort while getting in or off the chair. In the horizontal position a leg low position is thus possible (forward adjustment) or a head low position up to -10° (backward adjustment). This adjustment hence takes over 4 functions: head low position, leg low position, psoas position, standing up aid.

armrests

The armrests are made of high-quality integral foam. They can be swung away **(5)** and grant an optimal transfer of the patient from the bed to the chair and back.

legrests and footrests

There are 4 versions available: with fixed length or extendable by 11 cm – each with or without extendable and detachable footrest. In this manner careLine can be adapted to the body height and the requirements of each patient.

handle

The ergonomically shaped, large handle with rubber grip **(6)** is mounted to the backrest and serves also as shock protection.

cleaning

Definite shapes, functional coverings of the mechanic parts and disinfectable surfaces are designed for the hygienically conditions of hospitals. The seat cushion is removable. The surface underneath is easy to clean and prevents soiling of the mechanic components. **(7)**

accessories

Our accessories program accounts for all aspects of a possible application area (see page 10-11).

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adjustment function of the upper part, irrespective of the adjustment of the backrest and legrest

handle (accessory)

armrests can be swung away

removable seat cushion (for cleaning)

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Art. Nr. **4460121**

careLine fix with extendable legrest,
head bolster as accessory.

p/n **4460121**

careLine fix is not adjustable in height. The entrance height of the seat is 51-58 cm (depending on the adjustment of the seat angle). careLine considers the requirements and needs both of the patient / home resident and the care personnel. From the upright sitting position to the horizontal an ergonomic sitting and laying posture is assured. careLine can be adapted individually to your demands: You can have either a manual or electrical adjustment of the backrest and legrest. In addition 4 different legrests are available. By means of 4 double castors, the chair can easily and precisely be moved. All models have an inclinable seat to facilitate the work on the patient, e.g. a forward inclination serves as standing up aid and reduces therefore the effort of the patient while getting in or off the chair.

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careLine hv with extendable legrest and
electrical adjustment of the upholstered
surface. p/n 4470221

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Art.Nr. **4470221**

careLine hv with extendable legrest and electrical adjustment of the upholstered surface. Head bolster, upholstered armrests and side parts as accessories.

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careLine hv is hydraulically adjustable in height by 20 cm by a lever mounted on both sides. This facilitates the transfer of the patient when he cannot sit upright and assist the relocation. careLine considers the requirements and needs both of the patient/home resident and the care personnel. From the upright sitting position to the horizontal an ergonomic sitting and laying posture is assured. careLine can be adapted individually to your demands: You can have either a manual or electrical adjustment of the backrest and legrest. In addition 4 different legrests are available. By means of 4 double castors, the chair can easily and precisely be moved. All models have an inclinable seat to facilitate the work on the patient, e.g. a forward inclination serves as standing up aid and reduces therefore the effort of the patient while getting in or off the chair.

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